

Natus[®] EDS 3[™]


CSF External Drainage System

- Entirely disposable, complete system
- Simple setup
- Hydrophobic, hemorepellent Fortex membrane
- Laser accuracy
- Needle-free access
- Not made with natural rubber latex

CSF external drainage system designed with and for clinicians

With its ease-of-use, integrated pole clamp, Fortex membrane and accurate leveling from a unique laser leveling device the EDS 3 System is unequalled.

The EDS 3 System provides all this, plus upgraded stopcocks, pressure-resistant tubing, and a multi-functional drip chamber.

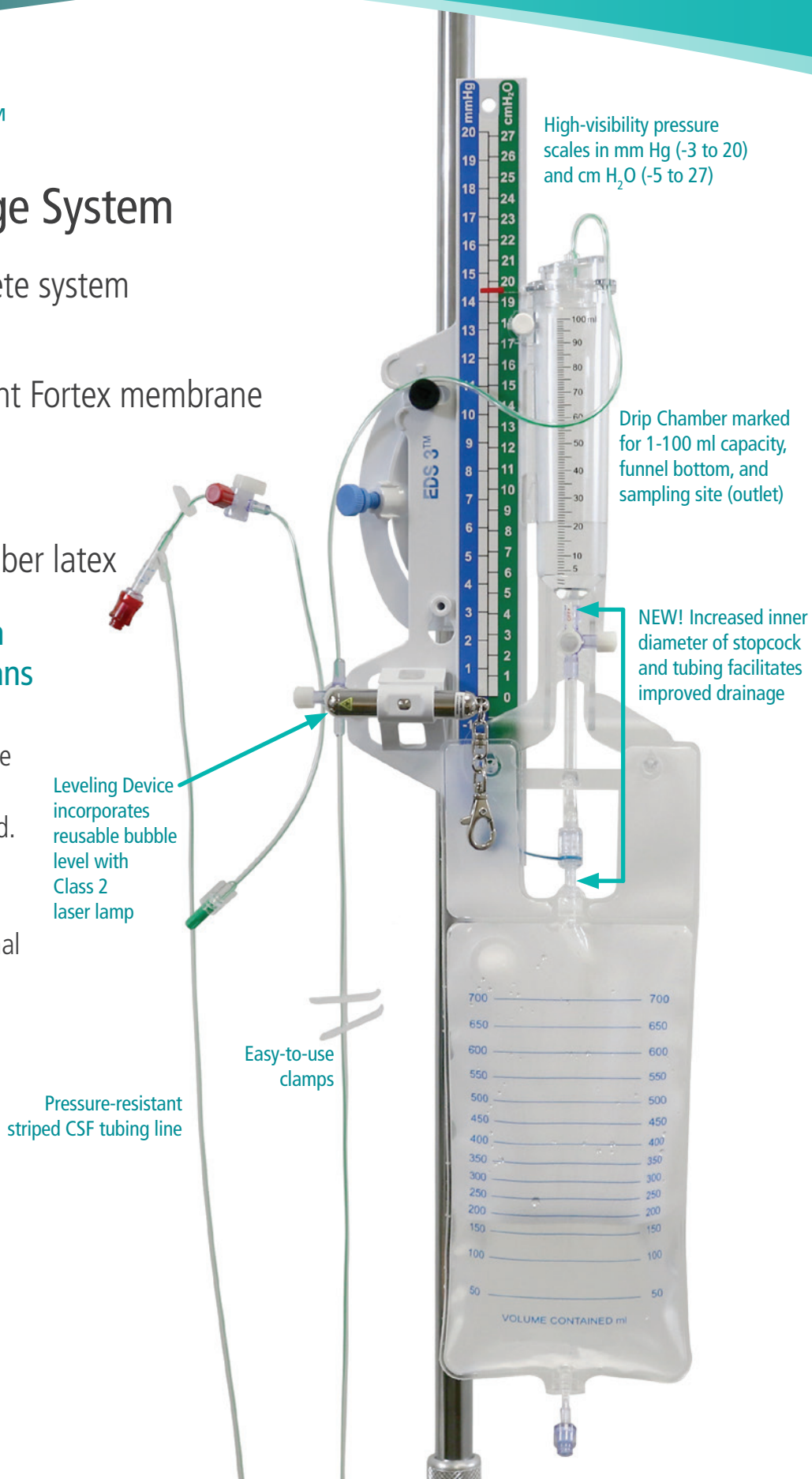

Easy Setup

1
Attach System
 to I.V. pole with
 Blue Screw




2
Insert
 leveling
 device




3
Level to the
 floor with
 Grey Screw



4
Press button
 to engage laser

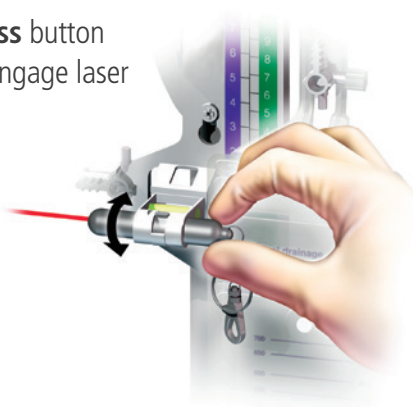

5
Adjust system height on
 the I.V. pole for proper zero
 reference with Blue Screw

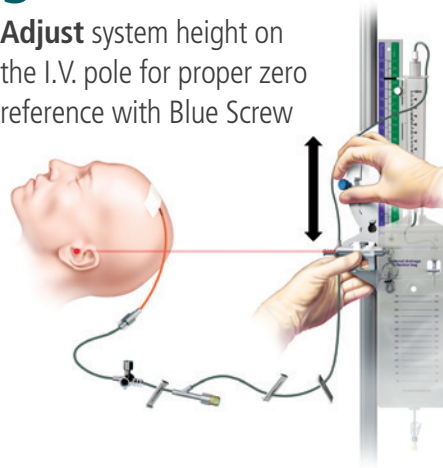

6
Adjust drip
 chamber height
 until black
 level is at the
 proper drainage
 resistance
 pressure with
 White Screw



Ordering Information

Catalog Number	Description
821730C	EDS 3 Drainage System
821731C	EDS 3 Drainage System no Ventricular Catheter
821738C	EDS 3 with Lumbar Catheter Kit II
821732C	EDS 3 Collection Bag Kit – 5 Bags
821733	EDS 3 Leveling Device



Natus Neuro
 3150 Pleasant View Road
 Middleton, WI 53562 USA
 Tel: 1-800-356-0007
 1-608-829-8500
 Fax: 1-608-829-8709
neuro.natus.com

To learn more about Natus Neuro products
 contact your sales representative.

US Customers Call: **1-800-356-0007**