


January 28, 2019

**Re: InVisus 5.3.2 software update**

Dear Valued Customer:

We are pleased to announce the release of the InVisus 5.3.2 software update. Below are some of the high-level changes in version 5.3.2:

- Preset Improvements with one touch access on the main screen.
  - Press the  symbol for quick access to presets
  - Frequency and maximum depth displayed
  - Increased maximum depths
  - General superficial, middle and deep presets added
- Free hand measurement (“contour”) can be done with finger or stylus.
  - Brighter color to measurements and text
- Confirmation pop up message appears when deleting a measurement.
- Color can be inverted when in Color Flow Mode for user preference.
- M-Mode is now displayed on the control wheel.
- CINE files can now go up to 128 frames

Customers are entitled to the update to version 5.3.2 at no charge. To download this software free-of-charge, please contact our Technical Support department for further information. Please include your InVisus model and serial number in any correspondence.

**Natus Technical Support Phone:** 1-800-356-0007 or 1-608-829-8500

**Natus Technical Support Email:** [MTS@natus.com](mailto:MTS@natus.com)

Natus Neuro is committed to developing the very best Neurodiagnostic products.

Thank you for being our healthcare partner and please feel free to contact your sales representative for further information.

Sincerely,



Adam Kornegay  
Senior Global Product Manager – EMG/Ultrasound  
Natus Neuro