

Please refer to Camino monitor and each individual catheter's quick start guide for technique details.

Parenchymal Bolt



REF: 110-4B
 OLM INTENSIVAMENTE TESTADO E APROBADO PARA
 USAR EN SU MONITOREO DE PRESION INTRACRANEAL.
 SOLO PARA USAR EN MONITOREO DE PRESION INTRACRANEAL.
 SOLO PARA USAR EN MONITOREO DE PRESION INTRACRANEAL.
 SOLO PARA USAR EN MONITOREO DE PRESION INTRACRANEAL.
 SOLO PARA USAR EN MONITOREO DE PRESION INTRACRANEAL.

1104B Catheter and Label



REF: 110-4BT
 INSTRUMENTO PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.

110-4BT Catheter and Label



CAMCABL

Ventricular Bolt*



MODEL: 110-4HM
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.

1104HM Catheter and Label



REF: 110-4HMT
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.
 PARA EL MONITOREO DE TEMPERATURA INTRACRANEAL.

110-4HMT Catheter and Label



CAMCABL

Subdural



REF: 110-4G
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.
 PARA MONITOREO DE PRESION INTRACRANEAL.

1104G Catheter and Label



CAMCABL

*When using ventricular bolt (1104HM or 1104HMT) catheters, an external drainage system is also required (i.e. Natus® EDS 3™, or other system).

Note: To connect to a bedside monitor, you will need the PMIOPM1 cable (included with each Camino monitor) in addition to ICP-XX or ICT-XX cables that are compatible with your facility's bedside monitor.

Helpful hints

Fiber optic catheters

- The fiber optic catheters (1104B, 1104BT, 1104HM, 1104HMT, 1104G and 1104L) are MR unsafe
- Extreme bending and/or kinks can impair the performance of the fiber optic Pressure Transducer
- If monitoring is continued for more than 5 days, placement of a new system under sterile conditions is recommended
- When removing the bolted monitoring setup from the patient, loosen the compression cap, remove the catheter, then remove the bolt; DO NOT remove the catheter and the bolt at the same time

Troubleshooting

The mean ICP value on the touch screen displays two dashes (- -)

- Make sure the cable is properly connected to the Camino ICP monitor
- If catheter is damaged, replace the catheter

Temperature value displays two dashes (- -)

- Ensure the catheter is ICP/temperature catheter (e.g. 1104BT, 1104HMT)
- If faulty or poor connection cannot be resolved, replace catheter and/or cable

Camino ICP monitor does not power On or Off when power button is pressed

- Disconnect AC adapter cord from Camino ICP monitor and remove battery
- Re-insert battery
- Reconnect AC adapter cord to Camino ICP monitor
- Press the power button

Unable to transfer trend data to USB drive/flash drive

- Insert a different brand of USB drive
- Insert a USB drive with a FAT16 or FAT32 formatted system

The Camino ICP Monitor is indicated for use by qualified neurosurgeons or neurointensivists for measurement of intracranial pressure and temperature. The EDS 3 CSF external drainage system is indicated for draining cerebrospinal fluid (CSF) and other fluids of similar physical characteristics as a means of reducing intracranial pressure and CSF volume when the insertion of a permanent, internal shunt is not indicated.

Making sense of the body's signals

©2022 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates. **023778 RevD**

natus[®]

Natus Medical Incorporated

[natus.com](https://www.natus.com)