Save the gonads

www.savethegonads.com

Manufactured in the USA by:
Fabricado en los Estados Unidos por:
Fabriqué dans l'USA par:
Hergestellt in den Vereinigten durch:
Fabricato negli USA da:

Save the Gonads, Ltd.
12055 Mosteller Rd. • Cincinnati, OH 45241
Phone: +1 513-385-8147
Registered Design No.: 89219-0001

natus

newborn care

Distributed by:
Distribuido por:
Distribuí par :
Vertrieb:
Distribuito da:

Natus Medical Incorporated
5900 First Avenue South • Seattle, WA 98108 USA
Phone: +1 650-802-0400 • Fax +1 650-802-6620
Email: customer_service@natus.com
Web: www.natus.com

Thank you for giving
a Small Life a Bigger Future!

P/N IFU 7/17

HOW TO USE AN STG

(Save the Gonads)

Every admission to your unit should have the appropriate size STG at their bedside.

The STG can be used for male or female babies. With the flip of the heart both sexes are protected. The heart remains upright for the female, to cover the ovaries. For the male, the heart is flipped downward to cover the testicles.

The STG is lead encased in a soft satin covering and can be easily cleansed with an alcohol wipe. For infection control reasons the STG is meant for individual patient use only. The STG may be reused repeatedly for a baby, however, the STG should not be shared with other babies.

The STG contains lead and needs to be disposed of properly. Remind parents that if they do keep this as a souvenir, that it must be kept out of the reach of children.

When used properly the STG significantly reduces radiation exposure to the gonads. It is a tool only to be used by qualified medical personnel.

Positioning
for Girl

Positioning
for Boy

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"Save the Gonads" is not responsible for misalignment of the STG, causing radiation exposure to the gonads.

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