

neoBLUE[®] compact LED phototherapy system



Provides intensive blue light
in a versatile and efficient design
for treating newborn jaundice

natus[®]

The neoBLUE compact LED Phototherapy System provides incredible performance and value with many user-selectable features

Meets AAP guidelines for intensive phototherapy¹

- **Intensity:** Features two intensity settings to switch between standard (15 $\mu\text{W}/\text{cm}^2/\text{nm}$) and intensive (35 $\mu\text{W}/\text{cm}^2/\text{nm}$) phototherapy
- **Spectrum:** Utilizes blue light-emitting diodes (LEDs) to emit blue light in the 450 - 470 nm spectrum, matching the peak absorption wavelength (458 nm) at which bilirubin is broken down²
- **Surface area coverage:** Exposes a large amount of the infant's skin to treatment



neoBLUE compact system positioned with suction cup feet on top of an incubator

Designed for multiple configurations

- Use the light independently by placing directly on top of an incubator
- Combine with the arm for attaching the pole-mount accessory of most incubators and radiant warmers
- Attach the light and arm to the roll stand and use for infants in a bassinet, open bed, incubator or radiant warmer



neoBLUE compact system with arm attached to the pole-mount on a radiant warmer

Smart arm design

- Arm-rotating joints and gooseneck provides multiple adjustment options with drift-free positioning
- Light and arm can be easily moved out of the way to attend to baby
- Nurses can easily attach and remove the light and arm at the bedside without tools




Safe

- neoBLUE LEDs do not emit significant ultraviolet (UV) light – reducing the potential risk of skin damage
- neoBLUE LEDs do not emit significant infrared (IR) light – reducing the potential risk of fluid loss



neoBLUE compact system positioned with arm and roll stand over a bassinet

Designed for convenience and ease-of-use

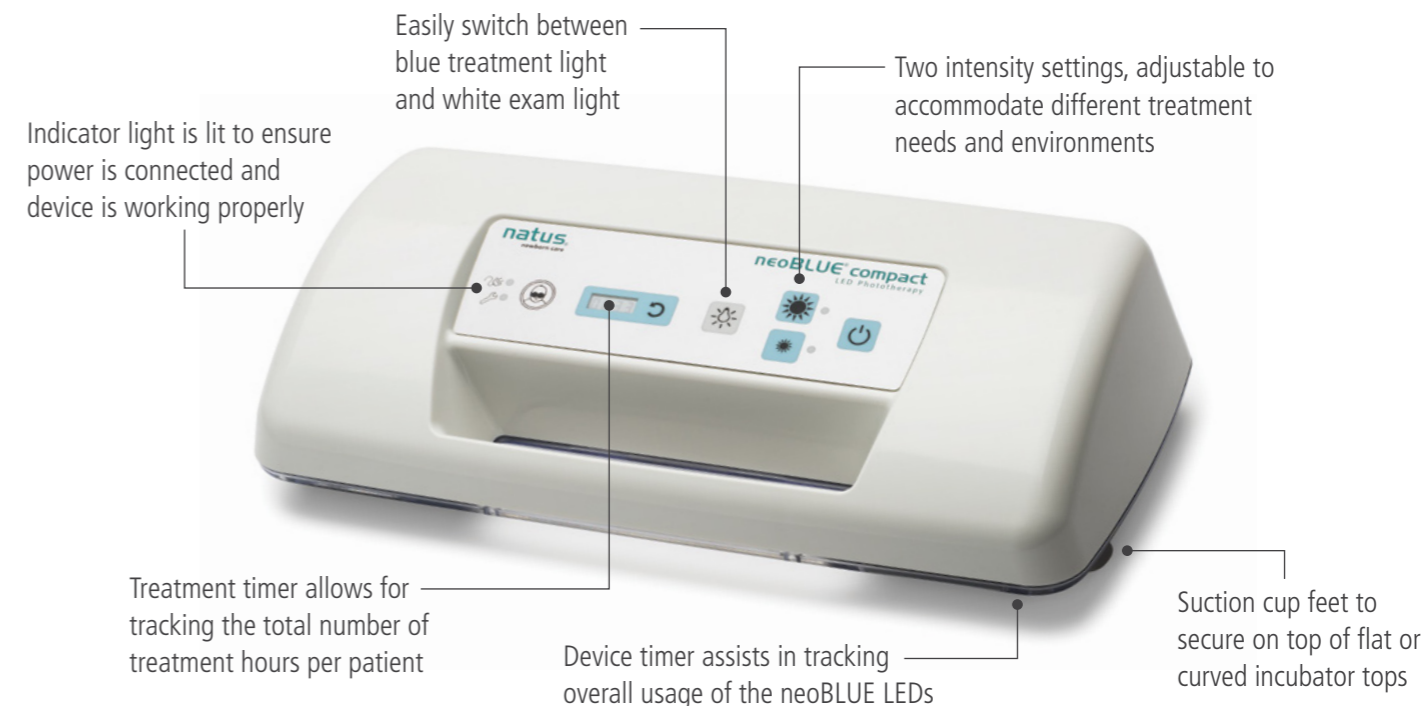

neoBLUE LEDs reduce costly and time-consuming bulb replacements by providing over 40,000 hours of use at high intensity³

Color-balanced for clinicians and families

- Twelve blue LEDs are mixed with a small amount of light from the white LEDs to soften the appearance of the blue treatment light while maintaining treatment efficacy
- Nurses and family sensitive to blue light will appreciate the softer baby blue appearance of the light



Brilliant white exam light


- Nine white LEDs provide bright illumination
- Neutral white light provides (true) color – ideal for general examination
- Perfect for monitoring babies, skin assessments, starting IVs, labs and basic exams
- Provides space-efficient solution with added functionality



neoBLUE[®] compact LED phototherapy system

Ordering Information

Item	Part #
neoBLUE compact LED Phototherapy System (includes light only)	019001
neoBLUE compact system w/arm (includes light and arm)	019011
Arm (available separately)	019030
Roll stand	019040

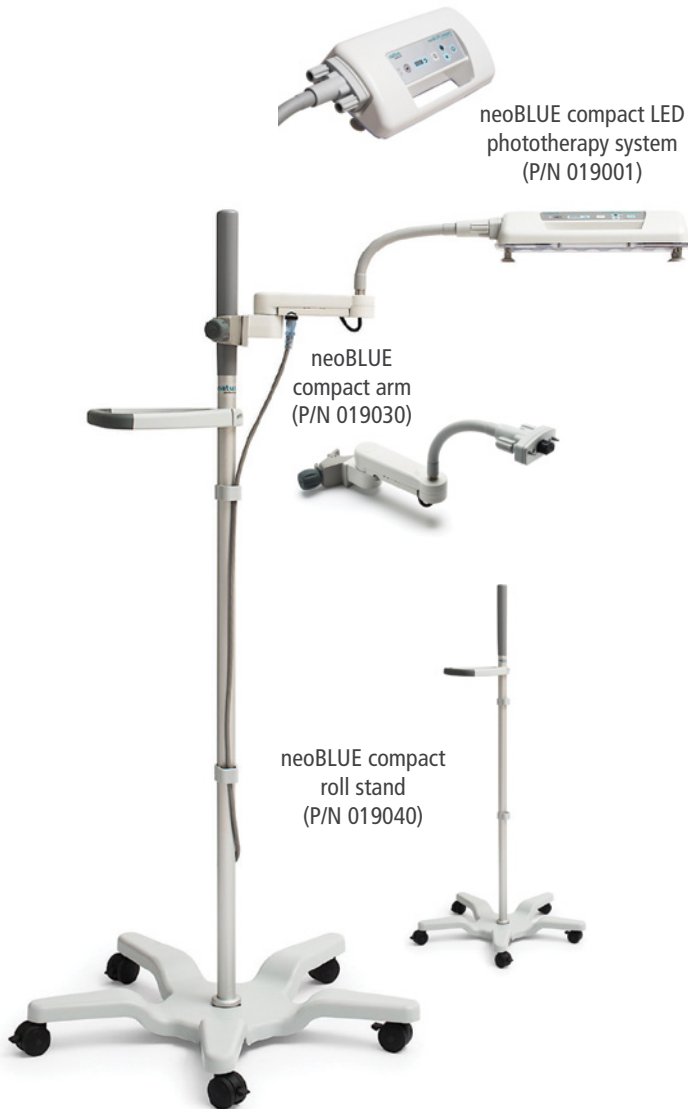

Technical Specifications

Light Source	Blue and White LEDs
Wavelength	Blue: Peak between 450 and 470 nm
Intensity	Peak intensity at 35 cm (13.75 in)
Factory setting	
Low	15 ±2 µW/cm ² /nm (total irradiance 1200 µW/cm ²)
High	35 ±2 µW/cm ² /nm (total irradiance 2800 µW/cm ²)
Adjustable setting	
Low	Approx. 10 - 35 µW/cm ² /nm
High	Approx. 30 - 55 µW/cm ² /nm
Variation in intensity over 6 hrs	< 1% (based on peak value within illumination area)
Effective surface area at 35 cm (13.75 in)	> 700 cm ² (108.5 in ²) Approx. 29 x 25 cm (11.4 x 9.8 in)
Intensity ratio	> 0.4 (minimum to maximum within effective surface area)
Heat output at 35 cm (13.75 in) over 6 hrs	< 1.7°C (3°F) warmer than ambient on mattress surface
LED life	> 40,000 hours of use at factory settings ³

White Exam Light

Color temperature	Approx. 4300K
Illuminance	Approx. 10,000 lux / 35 cm (13.75 in)

Electrical Mains

0.7A @ 100 - 240V~, 50 - 60 Hz

Safety

Leakage current	< 100 µA
Audible noise	< 40 dB

Weight

Light	< 1.2 kg (2.6 lb)
Arm	< 1.8 kg (4.0 lb)
Roll stand	< 10.9 kg (24 lb)

Roll Stand (with light and arm)

Height of lens from ground	Adjustable from approx. 1.24 - 1.57 m (49 - 62 in)
Center of lens from post	Adjustable up to approx. 61 cm (24 in) at fully-extended arm
Tilt adjustment of enclosure	Total rotation angle of arm's interface block approx. 55°
Clearance of base from floor	< 10.2 cm (4 in)
Base	Five legs with locking casters

Environmental

Operating temperature/humidity	5° - 35°C (41° - 95°F) / 10 - 90% non-condensing
Storage temperature/humidity	-30° - 50°C (-22° - 122°F) / 5 - 95% non-condensing
Altitude/atmospheric pressure	-1000 - 20,000 ft (50 - 106 kPa)

Regulatory Standards

IEC 60601-1: Editions 2 and 3
IEC 60601-2-50: Editions 1 and 2
IEC 60601-1-2: Editions 3 and 4 (EMC)
IEC 60601-1-6: 2010 (Usability)

Note: Specifications are subject to change without notice.

References

1 Subcommittee on Hyperbilirubinemia. American Academy of Pediatrics clinical practice guideline: Management of hyperbilirubinemia in the newborn infant 35 or more weeks of gestation. Pediatrics. 2004; 114(1):297-316.

2 Vreman HJ, et al. Light-emitting diodes: a novel light source for phototherapy. Pediatric Research. 1998; 44(5):804-809.

3 Actual results may vary based on environmental factors and adjustments to the intensity settings.

Healthcare solutions with one thing in mind. You.

©2021 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates. **017419 RevC**

natus[®]

Natus Medical Incorporated

natus.com