

Bio-logic® AuDX PRO FLEX®
Pump unit

Quick start guide

Doc. No.7-26-1378-EN/00
Part No.7-26-13708-EN




▶ Getting started

Place probe tip into the red probe cavity for the adult probe tip or the blue cavity for the infant probe tip
Turn the device on
Tap the tools icon
Select (tap) Functional Checks
Select Pump Unit Test

The **red lines** should be straight lines with no or just little oscillation or stairs effect.

The **green line** should ideally be horizontal (no pressure drop). With probe and cavity connected, some pressure drop is acceptable. If just testing the device with pressure outlet closed, there should be no pressure drop.

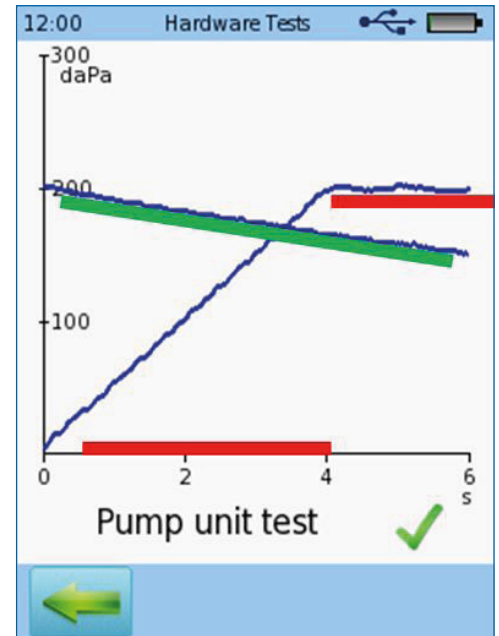

▶ Re-initialize pump unit

- Perform pump unit test
- After test is done, quickly tap on the graph 4 times
- Get "service menu"
- Press the 'x' button

