

Bio-logic® AuDX®
OAE screening
Quick start guide

Doc. No.7-26-1383-EN/00
Part No.7-26-13803-EN



▶ Turning On/Off, and charging the device




On/off button on right side



Power supply connector

▶ Start menu

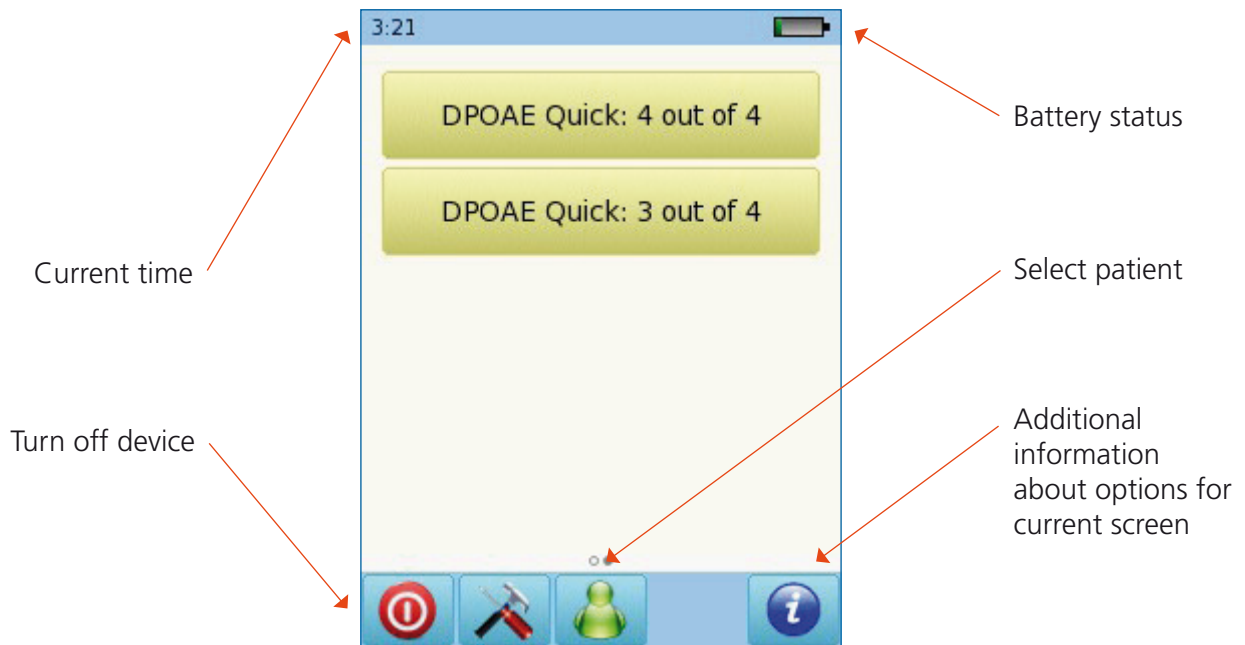

▶ Testing

Connect the probe to the white connector.

Select either DPOAE Quick: 3 out of 4 button or DPOAE Quick: 4 out of 4 button




Select the appropriate ear tip size and place it on the probe tip.


Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.

Select the test ear by pressing either the Right or Left button.

Select balloon mode to show balloons when the test is finished.

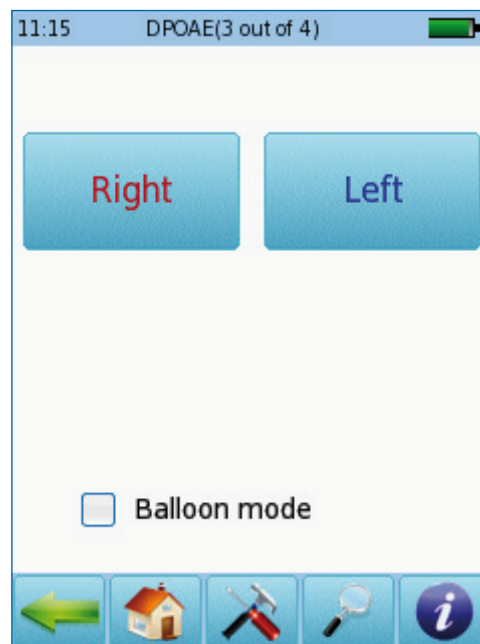

The device will check if the probe fit is correct prior to measurement. If the probe fit is correct (indicated by a green light), the test will start automatically.

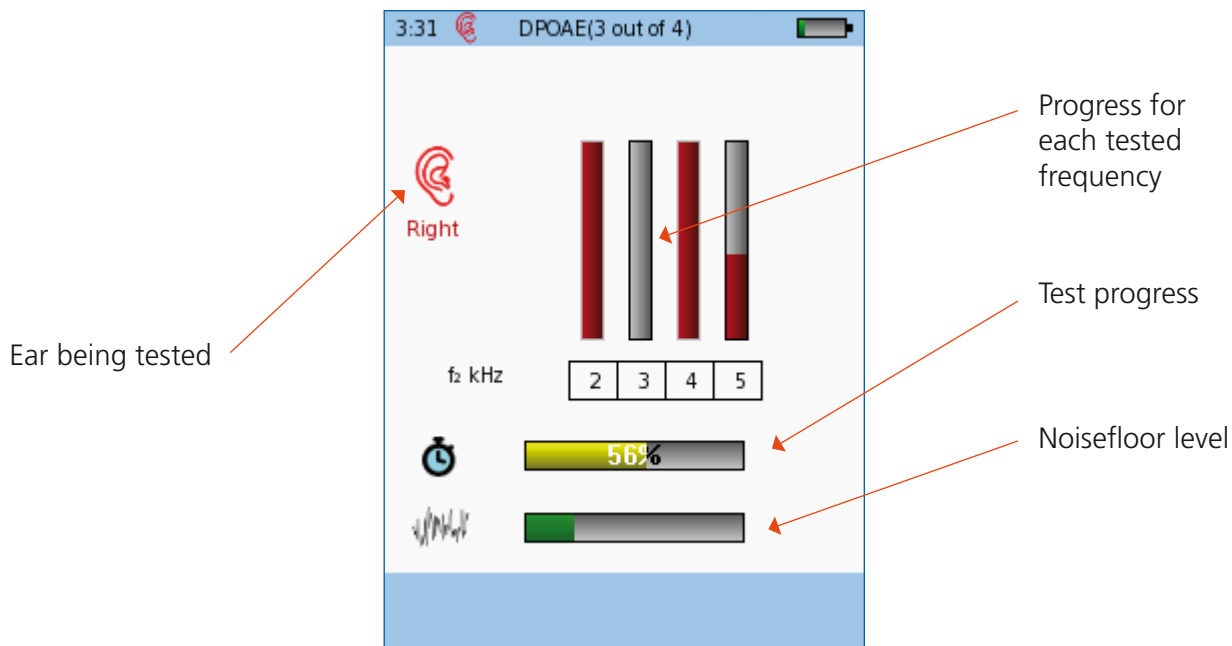

▶ Test results (bar graphs)

Overall screening test result



Green check marks indicate valid response. Red circles indicate no valid response

Red (right ear) or blue (left ear) bars show response amplitude. Grey bars show noise amplitude

Retest frequencies that did not pass

Print results to a label or to a pdf file

Test other ear

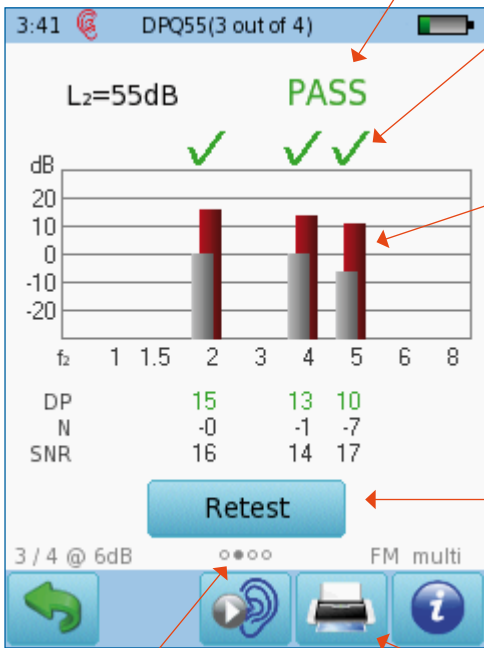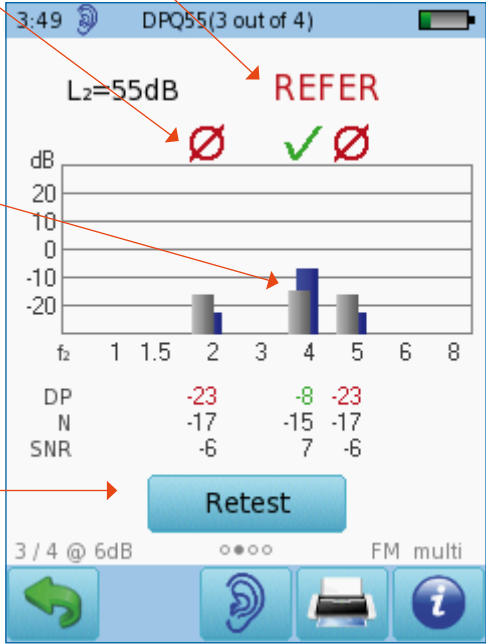

Note: These circles indicate that other screens are available to view. Swipe your finger right or left to get other views.

Switch to results of the other ear after both ears have been tested

▶ Test results (simple)

If the required criteria were met, the result will be **PASS**. If not, it will be **REFER**.

Test other ear

Print results to a label or to a pdf file or full page report on PC.

Re-test failed frequencies

Switch to results of the other ear after both ears have been tested

Quick Guide Information

Release date: 2017-01-11

Revision: 02

Valid from: Firmware Rev. 2.1, Mira PC Software Rev. 1.6.4

Applies to: Model: SOH100098, web description SOH00

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories.

Please contact your local distributor or PATH medical for further information.

The latest revision of the Quick Start Guide is available online at www.pathme.de/download.

Errors and omissions excepted.

Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2016 PATH medical GmbH

PATH medical GmbH
Landsberger Straße 65
82110 Germering
Germany
Tel: +49 89 800 76 502
Fax: +49 89 800 76 503
Internet: www.pathme.de