



Boost hygiene in immittance testing


The unique design of Madsen Zodiac makes cleaning and infection control an easy part of your immittance testing workflow

With new infection control standards, procedures and best practices, infection control is a top priority among hearing care professionals. But how can you prioritize hygiene in your clinic without sacrificing efficiency and your patient's overall experience?

Madsen Zodiac is the immittance testing solution from Natus that delivers control, confidence and efficiency in immittance testing. Madsen Zodiac streamlines the immittance testing process, including cleaning and infection control. Its unique design makes it easy to clean and reduces surface contact and patient-to-clinician contact. Hygiene and infection control become a natural part of your immittance testing workflow.



Madsen® Zodiac

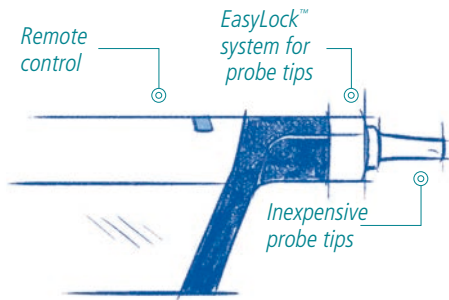

Simple to use and easy to clean, Madsen Zodiac enables you to prioritize hygiene and infection control in your clinic without sacrificing efficiency or your patient's experience.



"When I see a patient pass through the door and go into the Unit, I feel like I owe them a clean and relaxing environment. I want to make sure that when they leave, they don't carry an infection back home with them."

Maria De La Fosse, Ward Service Officer at Sandwell and West Birmingham Hospitals NHS Trust

Seven ways Madsen Zodiac helps promote hygiene in your clinic:

- Cleanable surfaces and robust materials comply with standard cleaning and disinfecting procedures and can withstand the long-term effects of harsh cleaning materials making the device easier to clean and maintain.
- Automated testing processes such as sequences and remote controls on the probes mean less physical contact with the device thereby reducing the potential for contamination.
- Fewer buttons on the Madsen Zodiac devices mean less chance for contamination. The Madsen Zodiac Quick Check version has dedicated hardware designed for screening. All excess buttons have been removed.
- Sealed buttons reduce the tiny hard-to-clean gaps and reduce the risk of damage from disinfecting liquid. The entire device can be wiped clean without the risk of damage.
- Shoulder strap is simple to fit and made from silicon-like material that is easier to clean than fabric. The shoulder pad and strap can be taken apart for thorough cleaning.
- Inexpensive probe tips and the EasyLock™ system mean you can change probe tips easily and often.
- Robust, extra-long cables are sealed to protect them from dust and debris and makes them easy to clean.

Built from the ground up with hearing care professionals just like you, Madsen Zodiac delivers the control, confidence and efficiency you need for successful immittance testing.

Madsen Zodiac is available in three versions:

- Quick Check
- Diagnostic
- Clinical

All three versions are available as stand-alone or PC-based solutions optimized for touch screen.

Contact your local Natus representative for a demonstration or visit hearing-balance.natus.com/zodiac

Healthcare solutions with one thing in mind. You.

©2020 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates. 7-26-4706-EN Rev03

natus

Natus Medical Incorporated

natus.com