


Quick start

▶ Turning On/Off and changing the device




▶ Start menu

Note: this screen (test options) can vary depending on your set up

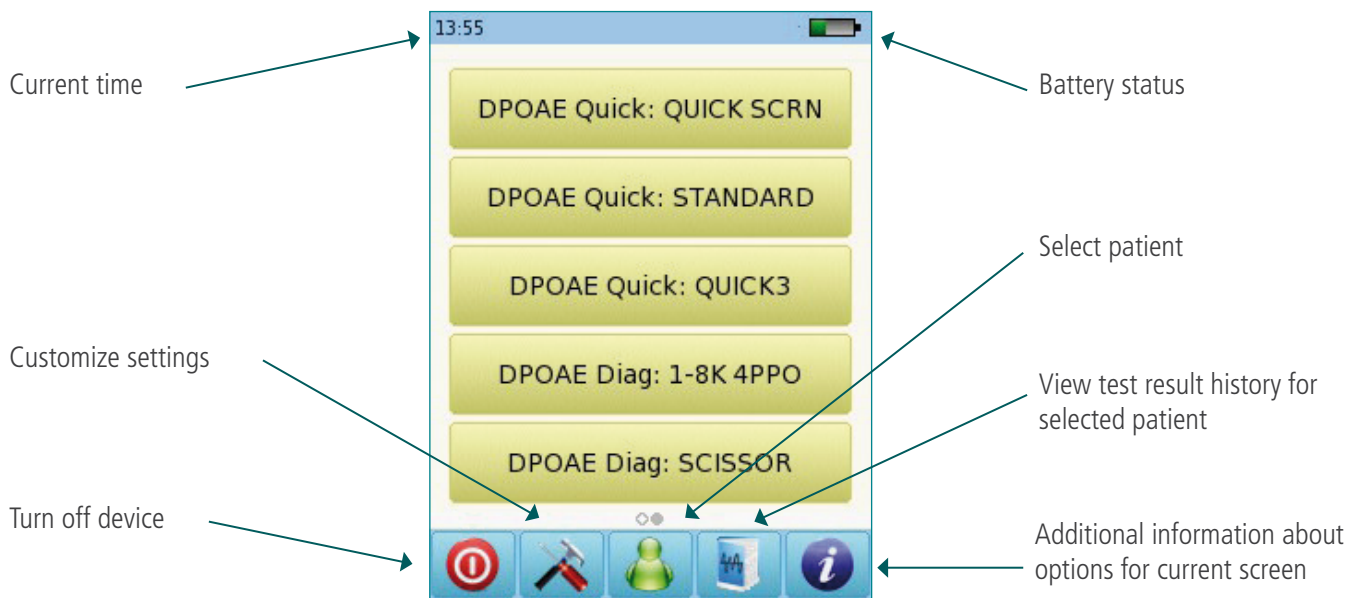


▶ Testing: Distortion Product Otoacoustic Emissions (DPOAE) Screening



Connect the probe to the red connector.
Select one of the Presets for DPOAE Screening

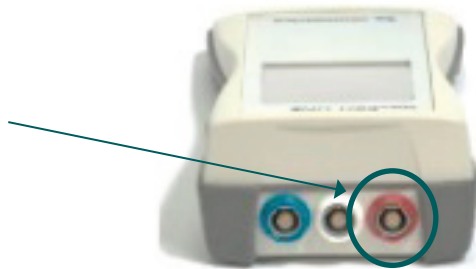


Select the appropriate ear tip size and place it on the probe tip.
Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.
Select the test ear by pressing either the Right or Left button.
Cartoon mode will start the test showing a cartoon instead of the progress bar.

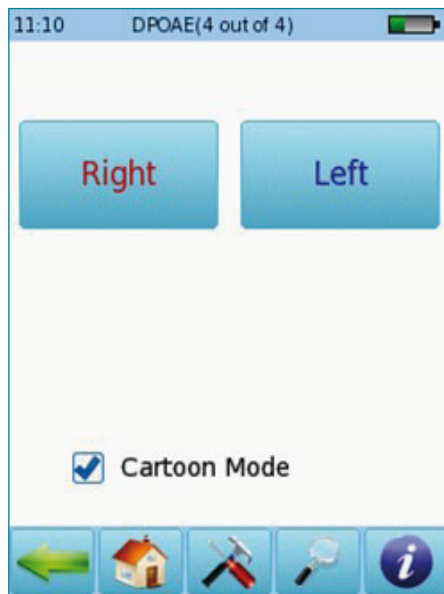

A calibration will be performed prior to measurement.
 A green light indicates successful calibration (below left).
 After successful ear probe calibration, the test will start automatically.

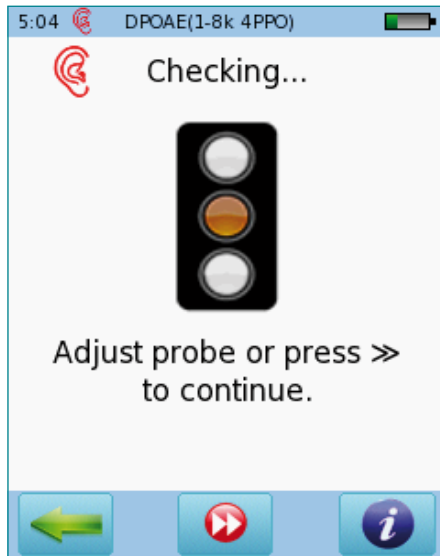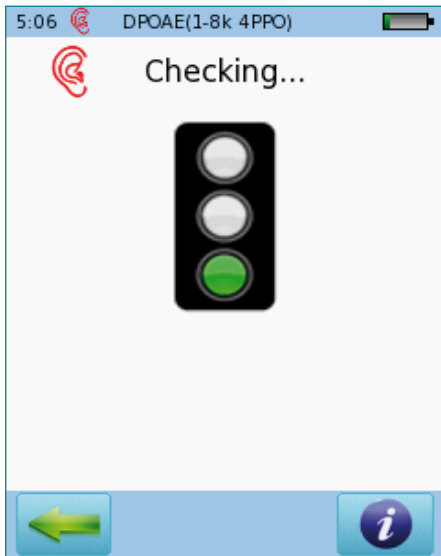

Probe fit override for patients with PE tubes or tympanic membrane perforation

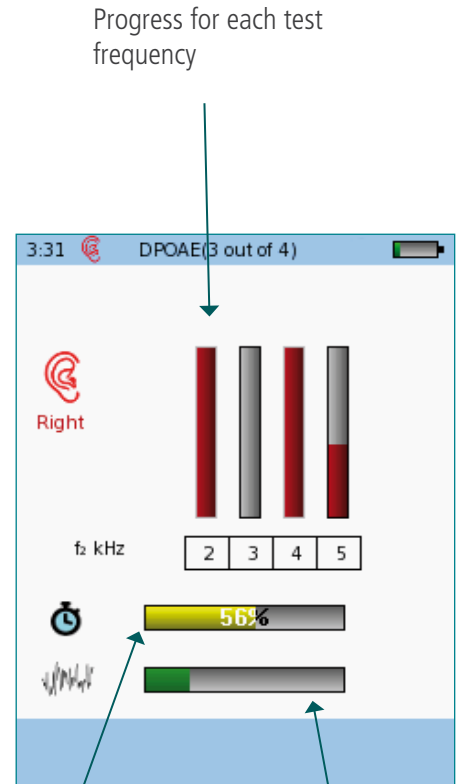

Overall test progress

Noise floor level

▶ DPOAE screening test results

Overall screening test result

Green check marks indicate valid response. Red circles indicate no valid response

Red (right ear) or blue (left ear) bars show response amplitude. Grey bars show noise amplitude

Retest frequencies that did not pass

Print results to a label or to a pdf file

Test other ear

Retest


Retest

Note: These circles indicate that other screens are available to view. Swipe your finger right or left to get other views.

Switch to results of the other ear after both ears have been tested


▶ DPOAE diagnostic / high resolution

Connect the probe to the red connector.
Select one of the Presets for DPOAE Diagnostic.

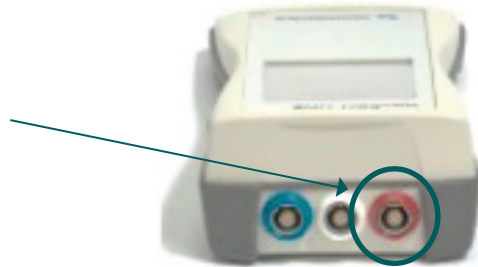


Select the appropriate ear tip size and place it on the probe tip.
Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.

Select the test ear by pressing either the Right or Left button. Select Cartoon mode to start the test showing a cartoon instead of the progress bar. This can be changed during the test by tapping on the plane icon at the bottom of the device screen.

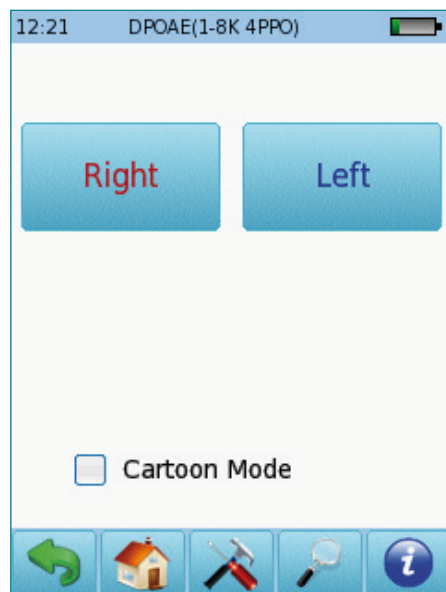


A calibration will be performed prior to measurement. A green light indicates successful calibration. After successful ear probe calibration, the test will start automatically. Please note that the presets for DPOAE diagnostics are HiRes, which allows testing frequencies in 10 Hz increments or up to 30 points per octave.

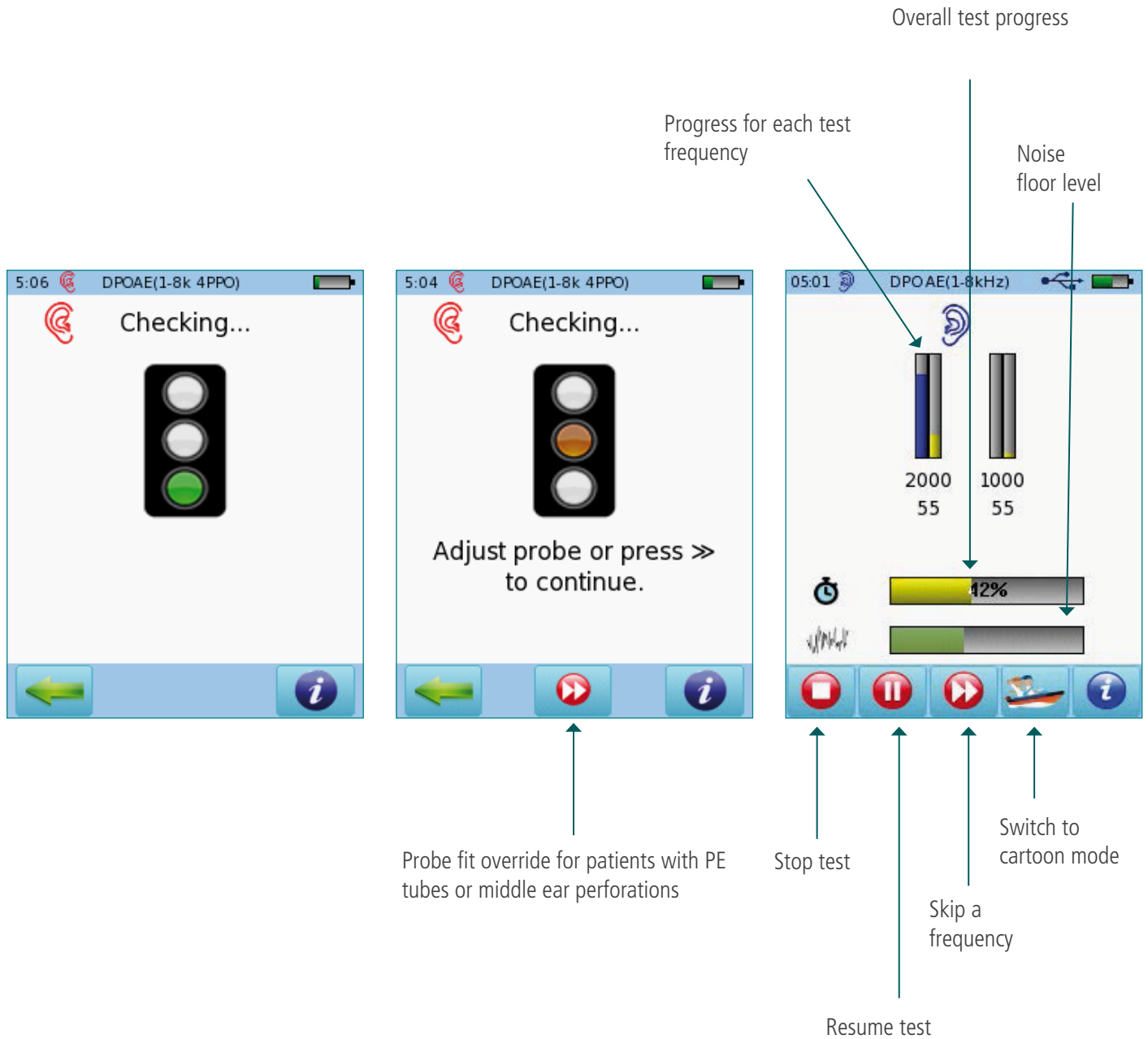


▶ DPOAE diagnostic test results

Green check marks indicate valid response

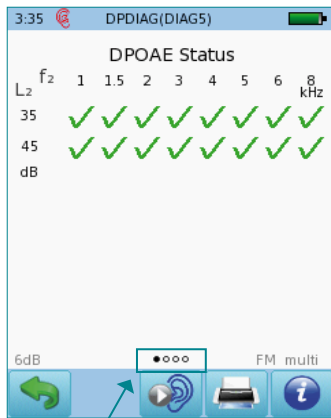

Note: These circles indicate that other screens are available to view. Swipe your finger right or left to get other views.



Red or blue bars indicate response, grey bars indicate noise

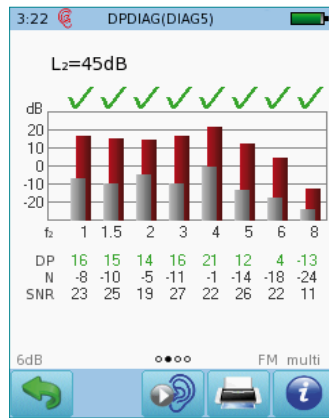

Print results to a label or to a pdf file

Outcome for each test sorted by f₂ and L₂.

Multilevel test results shown. Tap on screen to view details for each level tested.

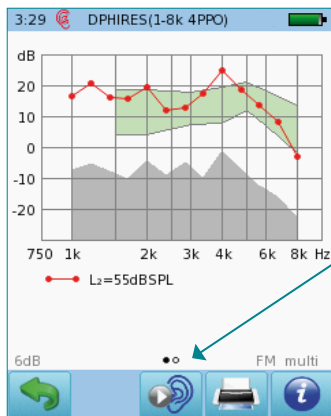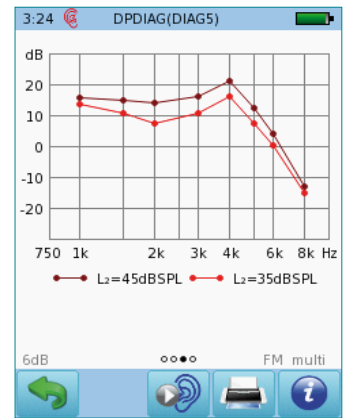

Test other ear

High resolution test results (4 PPO)

L ₂	f ₂	DPOAE	N	SNR
45	1000	15.7	-7.7	23.5
45	1500	14.9	-10.0	25.0
45	2000	13.8	-5.2	19.0
45	3000	15.9	-10.6	26.6
45	4000	20.9	-1.4	22.3
45	5000	12.1	-13.9	26.0
45	6000	4.1	-18.1	22.2
45	8000	-13.3	-24.5	11.2
35	1000	13.5	-9.3	22.8
35	1500	10.5	-14.2	24.7
35	2000	7.5	-6.2	13.7
35	3000	10.6	-13.5	24.2
35	4000	16.0	-4.0	20.0
35	5000	7.4	-17.7	25.1
35	6000	0.1	-20.5	20.6
35	8000	-15.4	-24.2	8.8

Numeric values shown in table format

Quick Guide Information

Release date: 2017-01-11

Revision: 02

Valid from: Firmware Rev. 2.1, Bio-link PC Software Rev. 2.0

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

Please contact your local Bio-logic distributor for further information.

Errors and omissions excepted.

Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2016 PATH medical GmbH

Legal manufacturer of the Bio-logic devices:

PATH medical GmbH

Landsberger Straße 65

82110 Germering

Germany

Tel: +49 89 800 76 502

Fax: +49 89 800 76 503

Internet: www.pathme.de



Healthcare solutions with one thing in mind. You.

©2020 Natus Medical Incorporated. All Rights Reserved. All product names appearing on this document are trademarks or registered trademarks owned, licensed to, promoted or distributed by Natus Medical Incorporated, its subsidiaries or affiliates. **7-26-1385-EN Rev01**

natus®

Natus Medical Incorporated

natus.com