

# Aurical® Otocam 300

## **Aurical Otocam 300 and the Video Otoscopy Module** Reference Manual

Doc. No. 7-50-1340-EN/08  
Part No. 7-50-13400-EN

CE

**natus**®

---

**Copyright notice**

© 2014, 2020 Natus Medical Denmark ApS. All rights reserved. ® Otometrics, the Otometrics Icon, Aurical, Madsen, HI-PRO 2, Otoscan, ICS and HORTMANN are registered trademarks of Natus Medical Denmark ApS in the U.S.A. and/or other countries.

**Version release date**

2020-05-11 (216990)

**Technical support**

Please contact your supplier.

---

# Table of Contents

<b>1</b>	<b>Introduction</b>	<b>5</b>
<b>2</b>	<b>Intended use</b>	<b>6</b>
2.1	Aurical Otocam 300 and the Otosuite Video Otoscopy module	6
2.2	Specula with cerumen management	6
2.3	About this manual	6
2.3.1	Safety	6
2.4	Typographical conventions	7
2.4.1	Navigating this manual	7
<b>3</b>	<b>Unpacking and Installing</b>	<b>8</b>
3.1	Unpacking	8
3.2	Storing	8
3.3	Assembling Aurical Otocam 300	8
3.4	Connecting Aurical Otocam 300	9
3.5	Switching Aurical Otocam 300 on and off	9
3.6	The Aurical Otocam 300 cradle	10
<b>4</b>	<b>Capturing and editing pictures</b>	<b>11</b>
4.1	Tips on how to use the specula	11
4.2	Tips on cerumen removal	11
4.3	Preparing for a session with a client	11
4.4	Viewing pictures from old Otocam 300 software	12
4.5	The main window	12
4.6	Capture mode	13
4.6.1	Capture functions	13
4.6.2	Capturing a picture	14
4.6.3	Switching modes with Aurical Otocam 300	15
4.7	Edit mode	15
4.7.1	Right-click functions	16
4.7.2	Adding markers to a picture	17
4.7.3	Removing markers	17
4.7.4	Adding comments to a picture	18
4.7.5	Assigning a picture to the other ear	18
4.8	Deleting pictures	18
4.9	Copying pictures to the clipboard	19
4.10	Exporting pictures	19
4.11	Live Otoscopy in PMM	19
4.12	PC shortcut keys	19
<b>5</b>	<b>Maintenance and Cleaning of Aurical Otocam 300</b>	<b>21</b>
5.1	Service and repair	21
5.2	Cleaning	21
5.2.1	Cleaning accessories	21
5.3	Maintenance	21
<b>6</b>	<b>Troubleshooting</b>	<b>23</b>
<b>7</b>	<b>Aurical Otocam 300 video otoscope standards and safety</b>	<b>24</b>
7.1	Warning notes	26

---

7.2	Manufacturer .....	28
7.2.1	Responsibility of the manufacturer .....	28
<b>8</b>	<b>Technical Specifications</b> .....	<b>29</b>
App. 0.1	Accessories .....	29
8.1	Device .....	29
8.2	Notes on EMC (Electromagnetic Compatibility) .....	31
<b>Index</b>	.....	<b>37</b>

# 1 Introduction



Aurical Otocam 300 is a video otoscope (VO) for visually inspecting and capturing pictures of the ear canal, the tympanic membrane or other such applications. Aurical Otocam 300 is used in connection with the Otosuite Video Otoscopy module to capture and edit pictures.

## **Easy to use**

AURICAL Aurical Otocam 300 is designed for optimal ease of use. The buttons on the camera provide you with quick operation without having to reach out for the mouse or the keyboard of your computer.

## **The Otosuite Video Otoscopy module**

With the Otosuite Video Otoscopy module you can capture and edit pictures from AURICAL Aurical Otocam 300.

You can use the Otosuite Video Otoscopy module as a module under Noah version 3.6.1 or higher, or directly as a module in Otosuite. When you use the Video Otoscopy module under Noah the captured pictures are integrated with other client data.

In order to easily identify and use the pictures, you can add markers and comments to the individual pictures, and use the Otosuite Report Manager function to print reports.

## 2 Intended use

### 2.1 Aurical Otocam 300 and the Otosuite Video Otoscopy module

#### Users

Audiologists, hearing instrument dispensers, ENT doctors and other trained personnel. Please note that local regulations may define users for video otoscopy differently. Local regulations must be complied with at all times.

#### Use

To visually inspect the ear canal and the tympanic membrane, and to capture and store pictures of the ear canal and the tympanic membrane, or other such applications.

#### Intended patient population

The intended patient population is all patient groups from pediatric through adulthood.

### 2.2 Specula with cerumen management

Please note that the use of specula with cerumen management may require special training in order to authorize personnel to carry out cerumen removal. These requirements are locally defined. Local regulations must be complied with at all times. Natus Medical Denmark ApS cannot be held responsible for unauthorized use of specula.

### 2.3 About this manual

This is your guide to using the Aurical Otocam 300 and the Otosuite Video Otoscopy module.

We strongly recommend that you read through the screen descriptions in this manual before you use Aurical Otocam 300 and the Otosuite Video Otoscopy module for the first time.

**Note** • *If you are using the Otosuite Video Otoscopy module with Noah, we recommend that you be familiar with the screens and functions provided in Noah.*

After you install Otosuite, you can find Otosuite manuals and related documentation on your PC. In the **Start** menu, open **Otosuite Manuals**, which contains an overview with links to all manuals.

#### 2.3.1 Safety

This manual contains information and warnings which must be followed to ensure the safe performance of Aurical Otocam 300 and the Otosuite Video Otoscopy module.

**Warning** • *Local government rules and regulations, if applicable, should be followed at all times.*

Safety information is stated where it is relevant, and general safety aspects are described in [Aurical Otocam 300 video otoscope standards and safety](#) ► 24

## 2.4 Typographical conventions

### The use of Warning, Caution and Note

To draw your attention to information regarding safe and appropriate use of the device or software, the manual uses precautionary statements as follows:

**Warning** • Indicates that there is a risk of death or serious injury to the user or patient.

**Caution** • Indicates that there is a risk of injury to the user or patient or risk of damage to data or the device.

**Note** • Indicates that you should take special notice.

### 2.4.1 Navigating this manual

Menus, icons and functions to select are shown in bold type, as for instance in:



Click the **Set options** icon on the toolbar or select **Tools > Options...**

## 3 Unpacking and Installing

**Note** • Make yourself familiar with the Otosuite Video Otoscopy Module, Aurical Otocam 300, and their functions and settings, before you start using them with a client.

To install and get started with Otocam 300 and the Otosuite Video Otoscopy module, follow the sequence below:

### Otosuite

- Install Otosuite on the PC. For Otosuite installation instructions, see the Otosuite Installation Guide, on the Otosuite installation medium.

### Aurical Otocam 300

- Unpack Aurical Otocam 300 ([Unpacking ► 8](#)).
- Assemble Aurical Otocam 300 ([Assembling Aurical Otocam 300 ► 8](#)).
- Connect Aurical Otocam 300 to the PC or to one of the USB connections on the rear of Aurical® Aud ([Connecting Aurical Otocam 300 ► 9](#)).

### 3.1 Unpacking

1. Unpack Aurical Otocam 300 carefully.

When you unpack Aurical Otocam 300 and accessories, it is a good idea to keep the packing material in which they were delivered. If you need to send Aurical Otocam 300 in for service, the original packing material will protect against damage during transport, etc.

2. Inspect the equipment for possible visual damage.  
If damage has occurred, do not put the device into operation. Contact your local distributor for assistance.
3. Check the packing list accompanying the package to make sure that you have received all necessary parts and accessories. If your package is incomplete, contact your local distributor.

### 3.2 Storing


If you need to store Aurical Otocam 300 before you put it into operation, follow the guidelines below:

- Store Aurical Otocam 300 and accessories in the boxes provided to protect the equipment from damage.
- Store Aurical Otocam 300 and accessories in a dry environment.

### 3.3 Assembling Aurical Otocam 300

Only the cradle needs to be assembled. Use the supplied Allen key, screws and washer to assemble the cradle as shown.



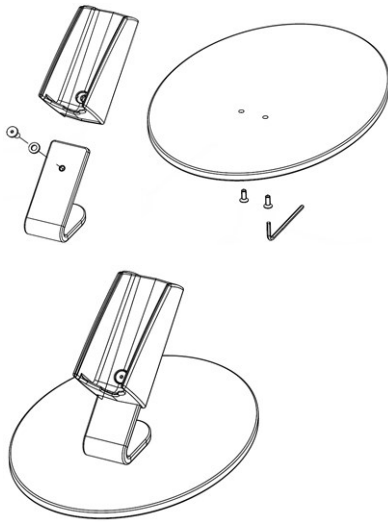

### 3.4 Connecting Aurical Otocam 300

1. Connect the USB cable of Aurical Otocam 300 to one of the computer's USB ports. Aurical Otocam 300 is powered through the USB connection to the PC.



If you are using Aurical Otocam 300 in connection with Aurical® Aud, you can alternatively connect Aurical Otocam 300 to one of the USB connections on the rear of Aurical® Aud.

2. Launch Otosuite and select the Otosuite Video Otoscopy module. Aurical Otocam 300 is automatically connected to the Otosuite Video Otoscopy module.

### 3.5 Switching Aurical Otocam 300 on and off

#### Switching on Aurical Otocam 300

1. Start up the computer.
2. Connect the USB cable of Aurical Otocam 300 to one of the computer's USB ports.
3. Launch Otosuite and select the **Video Otoscopy** module.

- If Otocam 300 is not placed in its cradle, the light beam is switched on.
- If Otocam 300 is placed in its cradle, the light beam is not switched on.

**Note** • In order for the pre-heater to heat the camera tip to body temperature, Aurical Otocam 300 should remain in the cradle (connected to a PC which is powered on) for minimum 5 minutes before the camera is used. The time should be extended if Aurical Otocam 300 has been stored in a cold environment.

**Warning** • Do not stare into the light beam, or point the light beam in the direction of other people's eyes. It can damage the eyes.

#### Switching off Aurical Otocam 300

To completely switch off Otocam 300, disconnect the USB cable from the computer.

## 3.6 The Aurical Otocam 300 cradle

The cradle is a multi-functional holder for your Aurical Otocam 300 when it is not in use. It controls the light source and the camera tip heating.

When Aurical Otocam 300 is placed in the cradle, the light source is turned off and the camera tip heating is turned on.

#### No condensation problems

In most cases the camera tip has a lower temperature than the air in the ear canal. This causes condensation, which reduces the image quality.

In Aurical Otocam 300 the built-in heater pre-heats the camera tip so that the temperature difference between the camera tip and the air inside the ear canal is minimal. The pre-heating function is activated when the Aurical Otocam 300 is plugged into the PC (which is powered on) and placed in its cradle. In order for the pre-heater to heat the camera tip to body temperature, Aurical Otocam 300 should remain in the cradle for approx. 5 minutes before the camera is used.

Therefore, for optimal operating conditions, place Aurical Otocam 300 in the cradle when you are not using it.

## 4 Capturing and editing pictures

### 4.1 Tips on how to use the specula

**Warning** • Be careful when you insert the speculum in the ear of the patient - there is a risk of damaging the wall of the ear canal and/or the tympanic membrane.

The specula are important elements in the hygienic aspect of video otoscopy. The specula must be disposed of after single use.

It is easy to snap the speculum onto and off the metal tip of the Aurical Otocam 300.

- To attach the speculum, press the speculum onto the metal tip until a click is heard.
- To detach the speculum, press on the rim of the speculum with your thumb until it is released from the metal tip.

### 4.2 Tips on cerumen removal

**Warning** • Be careful when you insert the speculum in the ear of the patient - there is a risk of damaging the wall of the ear canal and/or the tympanic membrane.

You can use specula specifically designed for cerumen removal with Aurical Otocam 300.

Using a speculum to remove cerumen has several advantages over other techniques.

- The lighted camera allows a full view of the area to be cleaned.
- If you use a cerumen removal speculum, it may not be necessary to apply any fluids in the ear canal. This makes the speculum approach useful and effective, even for cases with PE tubes, and for other situations where the tympanic membrane is not intact.

### 4.3 Preparing for a session with a client

1. Connect Aurical Otocam 300 to the PC.

If you are using Aurical Otocam 300 in connection with Aurical® Aud, you can alternatively connect Aurical Otocam 300 to one of the USB connections on the rear of Aurical® Aud.

2. Launch Otosuite.

The basic Otosuite functions are described in the OtosuiteUser Guide.

3. If needed, create a client.
4. Click the **Navigator** icon in the Otosuite toolbar to open the **Navigation Panel**.

## 4.4 Viewing pictures from old Otocam 300 software

If you have launched the **Video Otoscopy** module from Noah with a client selected, you can set up the **Video Otoscopy** module to show pictures captured in the old version of the Otocam 300 software (non-Otosuite). The screen will then show results, if available, from a previous session selected in Noah.

If these pictures were made in the old version of the Otocam 300 Software (non-Otosuite), they are read-only in the **Video Otoscopy** module.


1. Select **Tools > Configuration Wizard...**
2. Click on **Configure** next to **Video Otoscopy**.
3. Make sure that the path indicates the correct path to the old Otocam 300 pictures. If it does not, click the arrow to browse and select the correct path.

## 4.5 The main window

The **Video Otoscopy** screen is divided into three main sections.

### Before capturing pictures


If Aurical Otocam 300 is connected, the **Video Otoscopy** module opens up showing a live image.

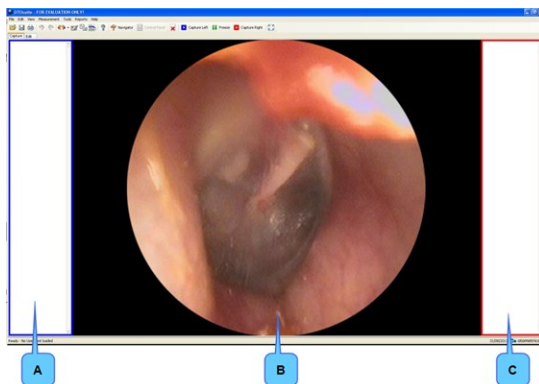

- A. Left Picture Panel
- B. Main work area
- C. Right Picture Panel

As default, the left **Picture Panel** is shown to the left of the screen. If you want to switch the right **Picture Panel** to the left side, select **Tools > Options > Video Otoscopy > General** and set **Picture Panel Viewing** to **Right-Left**.

After capturing pictures

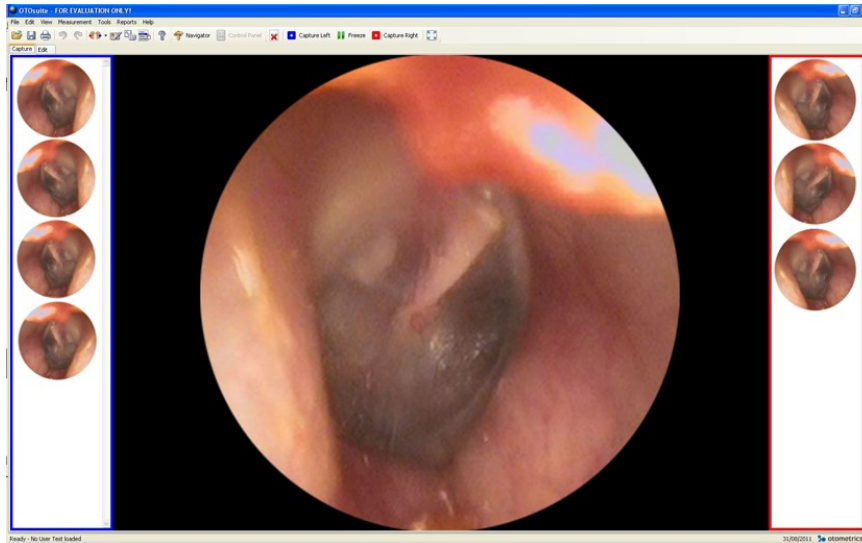

The left and right **Picture Panels** list the pictures you capture. Each picture is timestamped for easy identification.

## 4.6 Capture mode

In **Capture** mode you can visually inspect and capture pictures of the ear canal, the tympanic membrane or other such applications.


### 4.6.1 Capture functions


The Capture toolbar



Delete All Pictures	
	Deletes all pictures captured in this session.

Icon	Aurical Otocam 300	Shortcut	Capture
		L	Captures the picture as <b>Left Ear</b> .
		R	Captures the picture as <b>Right Ear</b> .

Freeze	
You can freeze a picture before capturing it as a picture.	
	<b>In regular screen mode</b> <ol style="list-style-type: none"><li>1. When an image is obtained, click the <b>Freeze</b> icon in the toolbar or press the <b>Spacebar</b>.</li><li>2. If needed, click the <b>Freeze</b> icon or press the <b>Spacebar</b> to unfreeze the image.</li></ol>
	<b>In Full Screen mode</b> <ol style="list-style-type: none"><li>1. When an image is obtained, press the <b>Spacebar</b> to freeze the image.</li><li>2. If needed, press the <b>Spacebar</b> to unfreeze the image.</li><li>3. Press <b>Esc</b> to return to regular viewing mode.</li></ol>

Full Screen mode	
	Displays a full screen view of the selected picture. <ol style="list-style-type: none"><li>1. When an image is obtained, press the <b>Spacebar</b> to freeze the image.</li><li>2. When a satisfactory image is obtained, press the <b>Right Ear</b> or <b>Left Ear</b> button on Otocam 300 to capture the image.</li><li>3. If needed, press the <b>Spacebar</b> to unfreeze the image.</li><li>4. Press <b>Esc</b> to return to regular viewing mode.</li></ol>


### 4.6.2 Capturing a picture

1. In Otosuite, click **Capture** in the **Video Otoscopy** section of the **Navigation Panel**. The **Video Otoscopy** module is launched and Aurical Otocam 300 is active.

**Warning** • Do not stare into the light beam, or point the light beam in the direction of other people's eyes. It can damage the eyes.

2. Press a speculum onto the tip of Aurical Otocam 300.
3. Insert the speculum on Aurical Otocam 300 in the ear of the client.

4. When a satisfactory image is obtained, press the **Right Ear** or **Left Ear** button on Aurical Otocam 300, or click the **Capture Left** or **Capture Right** icon in the toolbar.

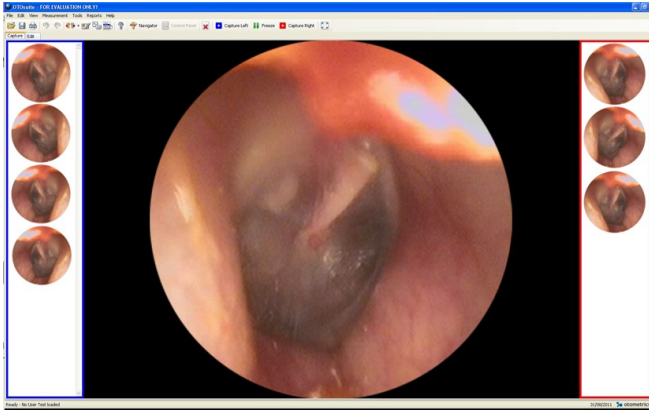

### 4.6.3 Switching modes with Aurical Otocam 300

If the **Video Otoscopy** module is in **Edit** mode, and you press the **Right Ear** or **Left Ear** button on Aurical Otocam 300, Otosuite will switch to **Capture** mode.

Press the **Right Ear** or **Left Ear** button again to capture a picture.

## 4.7 Edit mode


In **Edit** mode you can add markers and comments to the individual pictures.


If you have assigned a picture to the wrong ear when you captured it, you can assign it to the other ear.



### The Edit toolbar




#### Delete All Pictures

	Deletes all pictures captured in this session.
---	--


Markers	
	
<p>A marker consists of a circle or an arrowhead. When you place a marker on a picture, it will be identified by a marker identifier (the letters A to Z). Depending on the picture, the markers are white or black for easy visibility.</p> <p><i>Selecting a marker</i></p> <ul style="list-style-type: none"> <li>To select a marker, click on the desired marker in the <b>Edit</b> toolbar.</li> </ul> <p><i>Placing a marker on the picture</i></p> <ul style="list-style-type: none"> <li>To place the chosen marker on the picture, position the cursor at the point where you wish to place the marker and click once.</li> </ul>	


Eraser tool	
 	<p>The <b>Eraser</b> tool removes individual markers. This is useful if you wish to remove several markers.</p> <ul style="list-style-type: none"> <li>Select the <b>Eraser</b> tool and click on the marker.</li> <li>When you no longer want to use the <b>Eraser</b> tool, click on the <b>Pointer</b> tool or a <b>Marker</b> in the toolbar.</li> </ul>


Pointer tool	
	<p>The <b>Pointer</b> tool is selected as default.</p> <ul style="list-style-type: none"> <li>When you no longer want to use the <b>Eraser</b> tool or a <b>Marker</b> tool, click on the Pointer tool in the toolbar.</li> </ul>

### 4.7.1 Right-click functions


You can right-click on a picture in one of the **Picture Panels**, and in **Edit** mode also on the central picture.

Delete	
	<p>Deletes the selected picture.</p>


Edit	
	<p>Selects the picture for editing and displays it in the central work area.</p>


Copy to Clipboard	
	<p>Copies the selected picture to the clipboard. Markers, if there are any, will be included. You can paste the picture into other software programs.</p>



Swap Ear	
	If you have assigned a picture to the wrong ear when you captured it, you can assign it to the other ear. This will send the picture to the other <b>Picture Panel</b> .


### Right-click functions on the central picture

Remove All Markers	
	Right-click anywhere in the central picture and select <b>Remove All Markers</b> .


Remove Marker	
	Right-click on one marker in the central picture and select <b>Remove Marker</b> .

## 4.7.2 Adding markers to a picture

You can add markers to any picture.

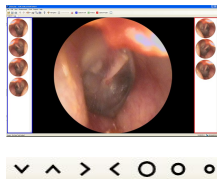

1. Click on the picture in the right or left **Picture Panel**. The picture is shown in the center of the screen.
2. To add a marker, click on one of the markers in the toolbar and click on the relevant area you wish to mark up on the central picture.  
The markers will be listed ranging from A to Z.

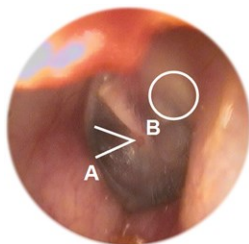

The corresponding picture in the **Picture Panel** will be marked with a **Comment** icon.

## 4.7.3 Removing markers

You can remove individual markers or all markers in a picture.

### Removing individual markers

You can remove individual markers in two ways:



- Right-click on the marker in the central picture and select **Remove Marker**.

or



- Click on the **Eraser** tool in the toolbar and click on the relevant marker or markers you wish to delete.


### Removing all markers



- Right-click on the central picture and select **Remove All Markers**.

## 4.7.4 Adding comments to a picture

You can add comments to any picture.

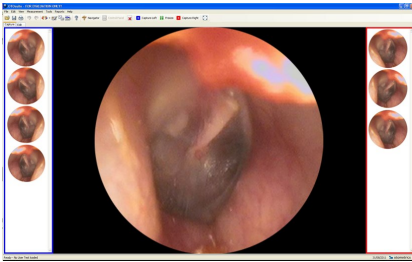

1. Click on the picture in the right or left **Picture Panel**. The picture is shown in the center of the screen.
2. To add a comment, write your comment in the **Comments** field below the central picture.



The corresponding picture in the **Picture Panel** will be marked with a **Comment** icon.

## 4.7.5 Assigning a picture to the other ear

If you have assigned a picture to the wrong ear when you captured it, you can assign it to the other ear.



- Right-click on the relevant picture in the right or left **Picture Panel** and select **Swap Ear**.

## 4.8 Deleting pictures

You can delete individual pictures or all pictures.

### Deleting individual pictures



- When you mouse over a picture in the right or left **Picture Panel**, a **Delete** icon is shown for that picture.

### Deleting all pictures



- Click the **Delete all pictures** icon in the toolbar.

## 4.9 Copying pictures to the clipboard



1. To copy a picture to the clipboard, right-click on the relevant picture and select **Copy to Clipboard**. Markers, if there are any, will be included.
2. You can paste the picture into other software programs.

## 4.10 Exporting pictures


- To export a picture to .jpg format, right-click on the relevant picture and select **Export**. You can export a picture with or without markers.

## 4.11 Live Otoscopy in PMM

You can open a live otoscopy window on top of the PMM window. If you use Aurical® FreeFit, this can be useful in order to monitor the placement of the probe tube, for example. It can also be useful for training clients.



Click the **Show Live Video Otoscopy** icon in the PMM tool bar to open the video otoscopy window.

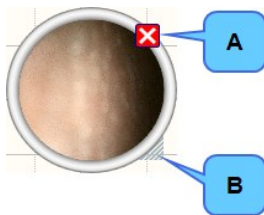

- A. Button for closing the window
- B. Grabber for resizing the window

## 4.12 PC shortcut keys

General shortcut keys are described in the Otosuite User Manual.

### Shortcut keys in Capture mode

Shortcut key	Function
Spacebar	Toggles the <b>Freeze</b> function.
L	Captures the picture as <b>Left Ear</b> .
R	Captures the picture as <b>Right Ear</b> .

Shortcut key	Function
<b>Alt + Enter</b> <b>Ctrl + Enter</b>	Enters <b>Full Screen View</b> (can also be done from toolbar icon, right-click menu or by double-clicking on picture).

#### Shortcut keys in Full Screen View mode

Shortcut key	Function
<b>Spacebar</b>	Toggles the <b>Freeze</b> function.
<b>L</b>	Captures the picture as <b>Left Ear</b> .
<b>R</b>	Captures the picture as <b>Right Ear</b> .
<b>Escape (Esc)</b>	Closes Full Screen View (can also be done from right-click menu or by double-clicking on picture).
<b>Alt + Enter</b> <b>Ctrl + Enter</b>	Closes <b>Full Screen View</b> .

## 5 Maintenance and Cleaning of Aurical Otocam 300

### 5.1 Service and repair

When you unpack Aurical Otocam 300 and accessories, it is a good idea to keep the packing material in which they were delivered. If you need to send Aurical Otocam 300 in for service, the original packing material will protect against damage during transport, etc.

For the sake of safety and in order not to void the warranty, service and repair of electro-medical equipment should be carried out only by the equipment manufacturer or by service personnel at authorized workshops. In case of any defects, make a detailed description of the defect(s) and contact your supplier. Do not use a defective device.

**Note** • *There are no user-serviceable parts inside the Otocam 300 housing.*

### 5.2 Cleaning

Never use sharp or pointed objects for cleaning!

**Warning** • *Do not clean the otoscope in an ultrasonic bath and do not gas-sterilize or autoclave the otoscope!*

1. Disconnect Aurical Otocam 300 from the computer.
2. Use a soft, slightly damp cloth with a small amount of mild detergent on it to clean the housing, camera head, plugs and cable.

**Caution** • *Do not allow any moisture inside the device!*

3. If the glass surface of the lens is very dirty, use a cotton pad with alcohol to clean it.
4. After cleaning, remove the cleaning agents thoroughly by wiping with a cloth dampened with pure, deionised water.
5. Finally, carefully dry all surfaces of Aurical Otocam 300 and the glass surface of the lens with a soft cloth.

#### 5.2.1 Cleaning accessories

##### **Specula**

Specula are disposable and therefore should not be cleaned or re-used.

There are no special requirements for the disposal of specula.

### 5.3 Maintenance

Aurical Otocam 300 requires no preventive maintenance except for cleaning and regular inspection of cable and plastic

housing of the device.

**Warning** • *For the sake of safety and in order not to void the warranty, service and repair of electro-medical equipment should be carried out only by the equipment manufacturer or by service personnel at authorized workshops. In case of any defects, make a detailed description of the defect(s) and contact your supplier. Do not use a defective device.*









## 6 Troubleshooting

Problem	Cause	Solution
The camera view area in the Video Otoscopy module is black.	Direct 3D is not rendered correctly by the graphics card.	Update the PC's graphics card driver.









## 7 Aurical Otocam 300 video otoscope standards and safety


This manual contains information and warnings, which must be followed to ensure the safe performance of the devices and software covered by this manual. Local government rules and regulations, if applicable, should also be followed at all times.

When the Video Otoscopy module is used in conjunction with a device (including devices other than those produced by Otometrics), make sure that all information and warnings in the documentation of the device are followed.

 ISO 15223-1 Symbol 5.1.1	<b>Manufacturer</b> Indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.
 ISO 15223-1 Symbol 5.1.3	<b>Date of manufacture.</b> Indicates the date when the medical device was manufactured.
 ISO 15223-1 Symbol 5.1.7	<b>Serial number</b> Indicates the manufacturer's serial number so that a specific medical device can be identified.
 ISO 15223-1 Symbol 5.1.6	<b>Catalog/product number</b> Indicates the manufacturer's catalogue number so that the medical device can be identified.
 IEC 60601-1 Table D.1 #20	<b>Type BF applied part</b> Complies with Type BF requirements of IEC 60601-1.
 93/42/EEC	<b>CE marking of conformity</b> Certification mark that indicates conformity with applicable regulations and directives for the European Economic Area.
	<b>Medical device</b>
 ISO 15223-1 Symbol 5.1.5	<b>Use-by date</b> Indicates the date after which the medical device is not to be used.

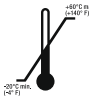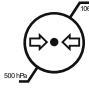

 <p>ISO 15223-1 Symbol 5.4.2</p>	<p>Do not reuse. Indicates a medical device that is intended for one use, or for use on a single patient during a single procedure.</p>
<p><b>R ONLY</b></p> <p>21 CFR Part 801. §801.109(b)(1)</p>	<p>Device is cleared for the US market as requiring a prescriptionUSA Code of Federal Regulations. 21 CFR Part 801. § 801.109(b)(1)</p>
 <p>ISO 15223-1 Symbol 5.4.3 and IEC 60601-1 Table D.1 #11</p>	<p>Consult instructions for use Indicates the need for the user to consult the instructions for use.</p>
 <p>IEC 60601-1 Table D.2 #10</p>	<p>Follow instructions for use</p>
	<p>Complies with Class II requirements of the safety standard IEC 60601-1:2005+AMD1:2012 and EN 60601-1:2006+A1:2013.</p>
	<p>MEDICAL - General Medical Equipment as to electrical shock, fire and mechanical hazards only in accordance with ANSI/AAMI ES60601-1:2005 + A1:2012, C1:2009/(R)2012 and A2:2010 (R)2012, CAN/CSA-C22.2 NO. 60601-1:14 , IEC 60601-1:2005+A1:2012.</p>
 <p>ISO 15223-1 Symbol 5.3.8</p>	<p>Humidity limitation Indicates the range of humidity to which the medical device can be safely exposed.</p>
 <p>ISO 15223-1 Symbol 5.3.7</p>	<p>Temperature limit Indicates the temperature limits to which the medical device can be safely exposed.</p>
 <p>ISO 15223-1 Symbol 5.3.9</p>	<p>Atmospheric pressure limitation Indicates the range of atmospheric pressure to which the medical device can be safely exposed.</p>

Made in China	Made in China Indicates that the device is manufactured in China.
Made in Thailand	Made in Thailand Indicates that the device is manufactured in Thailand. (Accessories only.)
 2012/19/EU	<p>Electronic equipment covered by the Directive 2012/19/EU of the European Parliament and of the Council of 4 July 2012 on waste electrical and electronic equipment (WEEE).</p> <p>All electrical and electronic products, batteries, and accumulators must be taken to separate collection at the end of their working life. This requirement applies in the European Union. Do not dispose of these products as unsorted municipal waste.</p> <p>You can return your device and accessories to Natus Medical Denmark ApS, or to any Natus Medical Denmark ApS supplier. You can also contact your local authorities for advice on disposal.</p> <p>See the full Natus Medical Denmark ApS WEEE statement below.</p>

### WEEE Statement

Natus is committed to meeting the requirements of the European Union WEEE (Waste Electrical and Electronic Equipment) Regulations 2014. These regulations state that electrical and electronic waste must be separately collected for the proper treatment and recovery to ensure that WEEE is reused or recycled safely. In line with that commitment Natus may pass along the obligation for take back and recycling to the end user, unless other arrangements have been made. Please contact us for details on the collection and recovery systems available to you in your region at [www.natus.com](http://www.natus.com)

Electrical and electronic equipment (EEE) contains materials, components and substances that may be hazardous and present a risk to human health and the environment when WEEE is not handled correctly. Therefore, end users also have a role to play in ensuring that WEEE is reused and recycled safely. Users of electrical and electronic equipment must not discard WEEE together with other wastes. Users must use the municipal collection schemes or the producer/importers take-back obligation or licensed waste carriers to reduce adverse environmental impacts in connection with disposal of waste electrical and electronic equipment and to increase opportunities for reuse, recycling and recovery of waste electrical and electronic equipment.

Equipment marked with the crossed-out wheeled bin is electrical and electronic equipment. The crossed-out wheeled bin symbol indicates that waste electrical and electronic equipment should not be discarded together with unseparated waste but must be collected separately.

## 7.1 Warning notes

- Do not stare into the light beam, or point the light beam in the direction of other people's eyes. It can damage the eyes.
- For continued protection against fire hazard, replace fuses with the same type and rating only.
- Do not use the instrument in the presence of flammable agents (gases) or in an oxygen-rich environment.
- No parts may be eaten, burnt, or in any way used for purposes other than the applications defined in the Intended Use section of this manual.
- For safety reasons and due to effects on EMC, accessories connected to the equipment's outlet fittings must be identical to the type supplied with the system.

- This class of equipment is allowed in domestic establishments when used under the jurisdiction of a health care professional.
- Unwanted noise may occur if the device is exposed to a strong radio field. Such noise may interfere with the performance of the device. Many types of electrical devices, e.g. mobile telephones, may generate radio fields. We recommend that the use of such devices in the vicinity of Aurical Otocam 300 be restricted.
- RF emissions from Aurical Otocam 300 are very low and are not likely to cause any interference in nearby electronic equipment. However, local devices placed in close vicinity of Aurical Otocam 300 may experience a negative effect or loss of functionality..
- It is recommended to install the unit in an environment that minimizes the amount of static electricity. For example, anti-static carpeting is recommended.
- We recommend that the device should not be stacked with other equipment or placed in a poorly ventilated space as this may affect the performance of the device. If it is stacked or placed adjacent to other equipment, make sure that the operation of the device is not affected.
- Before use, ensure that live video is provided on the screen.
- Before each use of Otocam 300, make sure there are no rough surfaces, sharp edges or protrusions.
- The distal tip may be warm.
- To prevent cross-infection or re-infection, do not use the device without a speculum mounted on the distal tip.
- To prevent cross-infection or re-infection, the speculum must be disposed of after single use.
- Accidental damage and incorrect handling can have a negative effect on the functionality of the device. Contact your supplier for advice.
- Aurical Otocam 300 is intended for use by audiologists and other trained health care professionals in visually inspecting the ear canal and tympanic membrane.
- Do not use the device for uses other than those described in the Intended Use section. For example, do not use the device for examination of nasal cavities, eyes or larynx.
- Do not store or operate Aurical Otocam 300 at temperatures and humidity exceeding those stated in Technical Specifications. Noncompliance can have negative effects on performance and/or cause degradation of device components.
- Be careful when you insert the speculum in the ear of the patient - there is a risk of damaging the wall of the ear canal and/or the tympanic membrane. Do not apply excessive force to the outer ear with the speculum.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and to the competent authority of the country or the EU Member State in which the user and/or patient is established



When you connect other electrical equipment to Aurical Otocam 300, remember that equipment that does not comply with the same safety standards as Aurical Otocam 300 can lead to a general reduction in the system's safety level. The equipment must comply with IEC 60950.



When selecting accessories connected to the Aurical Otocam 300, the following points must be considered:

- Use of connected equipment in a patient environment
- Proof that connected equipment has been tested in accordance with IEC60601-1 and/or IEC60601-1-1

Keep the unit away from liquids. Do not allow moisture inside the unit. Moisture inside the unit can damage the instrument and it may result in a risk of electrical shock to the user or patient.

## 7.2 Manufacturer

Natus Medical Denmark ApS  
Hoerskaetten 9, 2630 Taastrup  
Denmark  
☎ +45 45 75 55 55  
[www.natus.com](http://www.natus.com)

### 7.2.1 Responsibility of the manufacturer

The manufacturer is to be considered responsible for effects on safety, reliability, and performance of the equipment only if:

- All assembly operations, extensions, re-adjustments, modifications or repairs are carried out by the equipment manufacturer or personnel authorized by the manufacturer.
- The electrical installation to which the equipment is connected complies with EN/IEC requirements.
- The equipment is used in accordance with the instructions for use.

The manufacturer reserves the right to disclaim all responsibility for the operating safety, reliability and performance of equipment serviced or repaired by other parties.

## 8 Technical Specifications

### App. 0.1 Accessories

Accessory Name	Part Number
Desktop cradle	8-35-30800
Otosuite PC software	8-49-75800
Specula, normal (12 pcs)	8-62-42700
Specula, with cerumen management (12 pcs)	8-62-42710

### 8.1 Device

#### Type identification

Aurical Otocam 300 is type 1076 from Natus Medical Denmark ApS.

#### Video system

Sensor	0.3 inch CMOS Digital Image Sensor
Lens System	10 micro lenses with fixed focus with large depth of field
Sensor resolution	720 (H) x 720 (V) pixels
Frame rate	24 frames/second
Output signals	USB 2.0

#### Optical data

Minimum working distance	10 mm (0.4 inches)
--------------------------	--------------------

#### Preheater

The preheater is active when the Aurical Otocam 300 is plugged into the PC and placed in its cradle.

Preheater activation	Activated by magnet in cradle.
Preheater power	Heats camera distal tip to approx +5°C (9°F) above ambient temperature after 5 minutes of activation

#### Buttons

Freeze frame	Right/left ear
--------------	----------------

### Electrical data

Supply voltage through USB port

Input power USB 2.0, Max. 500 mA.

Light source 2 LEDs, fibre-optic light guide

USB plug USB, type A (LED power supply/Camera/Pre-heating)

Switch activated by magnet contact used to switch between pre-heating and light.

### Mechanical data

Cable length 2700 mm (8.86 feet)

Length without cable 170mm ( 6.7 inches)

Weight including cable 250 g (8.8 ounces)

Largest diameter 45 mm (1.8 inches)

Distal diameter max. 3.4 mm (0.134 inches)

Total weight 1300 g

### Service life

Expected Service life 5 years

### Storage environment

Temperature -20°C to +60°C (-4°F to +140°F)

Relative humidity <90%, non-condensing

Air pressure 500 hPa to 1060 hPa

### Operating environment

Temperature +10°C to +30°C (+50°F to +86°F)

Air humidity 30% to 75%, non-condensing

Air pressure 600 hPa to 1060 hPa

### Essential performance

Aurical Otocam 300 has no essential performance.

### Standards

Patient Safety IEC 60601-1:2005+AMD1:2012 and EN 60601-1:2006+A1:2013

CAN/CSA-C22.2 NO. 60601-1:14

Class II; applied part Type BF; IPX0

Endoscopic equipment:

IEC 60601-2-18:2009 and EN 60601-2-18:1996 + A1:2000

EMC

IEC 60601-1-2:2007 and EN 60601-1-2:2007

IEC 60601-1-2:2014 and EN 60601-1-2:2015

## 8.2 Notes on EMC (Electromagnetic Compatibility)

- Aurical Otocam 300 is part of a medical electrical system and is thus subject to special safety precautions. For this reason, the installation and operating instructions provided in this document should be followed closely.
- Portable and mobile high-frequency communication devices, such as mobile phones, may interfere with the functioning of Aurical Otocam 300.

### IEC 60601-1-2:2014 and EN 60601-1-2:2015

Guidance and manufacturer's declaration - electromagnetic emissions for all equipment and systems		
Aurical Otocam 300 is intended for use in the electromagnetic environment specified below. The user of Aurical Otocam 300 should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR11	Group 1	Aurical Otocam 300 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR11	Class B	Aurical Otocam 300 is suitable for use in all environments, including domestic environments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacturer's declaration - electromagnetic immunity for all equipment and systems			
Aurical Otocam 300 is intended for use in the electromagnetic environment specified below. The user of Aurical Otocam 300 should ensure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 8 kV contact +/- 2 kV, +/- 4 kV, +/- 8 kV, +/- 15 kV air	+/- 8 kV contact +/- 2 kV, +/- 4 kV, +/- 8 kV, +/- 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	+/- 1 kV for input/output lines	+/- 1 kV for input/output lines	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	No relevant ports that could be affected	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

## 8 Technical Specifications

Guidance and manufacturer's declaration - electromagnetic immunity - for equipment and systems within Professional Healthcare use environment			
Aurical Otocam 300 is intended for use in the electromagnetic environment specified below. The user of Aurical Otocam 300 should ensure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 V rms 150 kHz to 80 MHz	3 V rms 150 kHz to 80 MHz	
	6 V rms IISM Bands and Amateur	6 V rms IISM Bands and Amateur	
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m 80 MHz to 2.7 GHz	
Proximity fields from RF wireless communications IEC 61000-4-3	27 V/m 385 MHz	27 V/m 385 MHz	Separation distance between any electronic parts of Aurical Otocam 300 and any RF wireless communication equipment must be more than 30 cm (11.8 inches).  <b>Note:</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.
	28 V/m 450 MHz,	28 V/m 450 MHz,	
	9 V/m 710 MHz, 745 MHz, 780 MHz	9 V/m 710 MHz, 745 MHz, 780 MHz	
	28 V/m 810 MHz, 870 MHz, 930 MHz,	28 V/m 810 MHz, 870 MHz, 930 MHz,	
	28 V/m 1720 MHz, 1845 MHz, 1970 MHz	28 V/m 1720 MHz, 1845 MHz, 1970 MHz	
	28 V/m 2450 MHz,	28 V/m 2450 MHz,	
	9 V/m 5240 MHz, 5500 MHz, 5785 MHz	9 V/m 5240 MHz, 5500 MHz, 5785 MHz	

### IEC 60601-1-2:2007 and EN 60601-1-2:2007

Guidance and manufacturer's declaration - electromagnetic emissions for all equipment and systems		
Aurical Otocam 300 is intended for use in the electromagnetic environment specified below. The user of Aurical Otocam 300 should ensure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR11	Group 1	Aurical Otocam 300 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.



RF emissions CISPR11	Class B	Aurical Otocam 300 is suitable for use in all environments, including domestic environments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

**Guidance and manufacturer's declaration - electromagnetic immunity for all equipment and systems**

Aurical Otocam 300 is intended for use in the electromagnetic environment specified below. The user of Aurical Otocam 300 should ensure that it is used in such an environment.


Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 6 kV contact +/- 8 kV air	+/- 6 kV contact +/- 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	+/- 1 kV for input/output lines	+/- 1 kV for input/output lines	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

**Guidance and manufacturer's declaration - electromagnetic immunity - for equipment and systems that are NOT life-supporting**

Aurical Otocam 300 is intended for use in the electromagnetic environment specified below. The user of Aurical Otocam 300 should ensure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
---------------	-------------------------	------------------	--

## 8 Technical Specifications

Conducted RF IEC 61000-4-6	3 V rms 150 kHz to 80 MHz	3 V rms 150 kHz to 80 MHz	<p>Portable and mobile RF communications equipment should be used no closer to any part of Aurical Otocam 300, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance:</p> $d = 1.2\sqrt{P}$ $d = 2.3\sqrt{P}$ <p>where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with this symbol:</p> 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m 80 MHz to 2.5 GHz	
<p><b>Note 1:</b> At 80 MHz and 800 MHz the separation distance for the higher frequency range applies.</p> <p><b>Note 2:</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p> <p>a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which Aurical Otocam 300 is used exceeds the applicable RF compliance level above, the Aurical Otocam 300 should be observed to verify normal operation. If abnormal performance is observed, additional measures might be necessary, such as reorienting or relocating Aurical Otocam 300.</p> <p>b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.</p>			

### Recommended separation distances between portable and mobile RF communications equipment and Aurical Otocam 300

The Aurical Otocam 300 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Aurical Otocam 300 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Aurical Otocam 300 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3

100	12	12	23
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance <math>d</math> in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p><b>Note 1:</b> At 80 MHz and 800 MHz the separation distance for the higher frequency range applies.</p> <p><b>Note 2:</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			



# Index

## A

### Adding

- comments to picture 18
- markers to picture 17

### Assembly

- OTOcam 300 8

## C

### Capturing

- functions 13
- picture 14

### Cerumen removal 11

### Cleaning 21

### Comments

- adding 18

### Connecting OTOcam 300 9

### Copy to clipboard 19

### Cradle

- assembly 8
- description 10

## D

### Deleting pictures 18

### Device type 29

### Disposal of specula 21

## E

### Editing

- adding markers 17
- functions 15
- removing comments 18
- removing markers 17

### Eraser tool 15

### Exporting pictures 19

## F

### Freeze 13

### Full screen view 13

## I

### Icons

- Capture Left 13
- Capture Right 13
- Delete 13, 15

### Eraser tool 15

### Freeze 13

### Full screen view 13

### Pointer tool 15

### Installing

- OTOcam 300 8

### Intended use 6

### Introduction 5

## M

### Main window

- Video Otoscopy 12

### Maintenance 21

### Manufacturer 28

### Markers 15

- adding 17
- removing 17

## O

### OTOcam 300

- assembly 8
- cerumen removal 11
- connecting 9
- cradle 8, 10
- device type 29
- introduction 5
- maintenance and cleaning 21
- safety 24
- storing 8
- switching on/off 9
- unpacking 8
- unpacking and installing 8
- using specula 11

## P

### Pictures

- adding comments 18
- adding markers 17
- capturing 14
- copying to clipboard 19
- deleting 18
- exporting 19
- removing markers 17
- swapping ear 18
- viewing old OTOcam pictures 12

Pointer tool 15  
Preparing for a session 11

## **R**

Removing markers from picture 17  
Right-click functions 16

## **S**

Safety 24  
Screens  
    Video Otoscopy main window 12  
Service and repair 21  
Shortcut keys 19  
Specifications 29  
Specula  
    cerumen removal 11  
    disposal 21  
    how to use 11  
Storing the device 8  
Swap ear 18  
Switching OTOcam 300 on/off 9

## **T**

Technical specifications 29  
    video system 29

## Toolbar

    capture 13  
    edit 15

## **U**

Unpacking and installing  
    OTOcam 300 8  
Unpacking the device 8

## **V**

Video Otoscopy module  
    main window 12  
    PC shortcut keys 19  
    right-click functions 16  
Video system  
    technical specifications 29

## **W**

Warning notes 26