

## Installation of SoundHub 100



**Note** • Do not connect the SoundHub to the PC before you install the OTOSuite Software.




1. Connect the accessories to the SoundHub unit as shown in the figure.
2. Install the OTOSuite Software.
3. Use the supplied USB cable to connect the SoundHub to a USB port of a PC.

**Note** • If the SoundHub is connected to the PC via a USB hub, the hub must be self-powered (have a separate power supply).

## Configuration of OTOSuite for SoundHub 100\*

When running the OTOSuite Configuration Wizard for the Counseling and Simulations module:

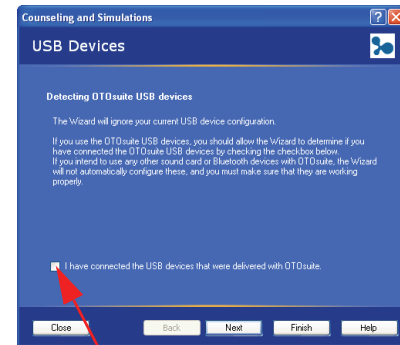


Do NOT check this, and press "Next"

\*) Continue configuration overleaf ▷.

## Configuration of OTOSuite for SoundHub 100\*

When running the Configuration wizard for either Counseling and Simulations or PMM, make the following sound card selections:

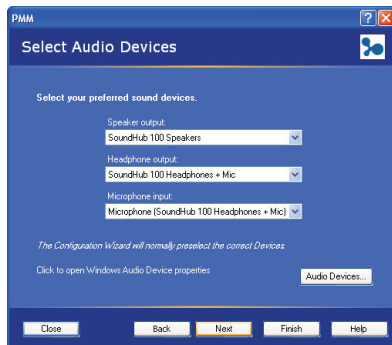

\*) Continued from overleaf.

### Copyright notice

No part of this installation guide or program may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of GN Otometrics A/S.

### Copyright© 2011, GN Otometrics A/S

Printed in Denmark by GN Otometrics A/S, Denmark  
All information, illustrations, and specifications in this manual are based on the latest product information available at the time of publication.

GN Otometrics A/S reserves the right to make changes at any time without notice.

### Version release date

1 September 2011

GN Otometrics A/S, Hoerskaetten 9,  
DK-2630 Taastrup, Denmark  
Phone: +45 45 75 55 55

### Technical support

Please contact your supplier.

# SoundHub 100 Installation Guide



CE  
0459

Part no. 7-50-07600-EN  
Doc no. 7-50-0760-EN/03