## Installation Guide

Installation - English	2
Installation - Deutsch	4
Instalación - Español	6
Installation - Français	8
Installazione - Italiano	10
Instalação - Português	12
Installatie - Nederlands	14
Installation - Dansk	16
Installation - Svensk	18
Installasjon - Norsk	20
Asennus - Suomi	22
Kurulum - Türk	24
Установка — русский	26
安装 - 中文	28
インストラクション - 日本語	30
نصب - فارسي	32
التثبيت - اللغة العربية	34

# Installation - English

## AccuLink documentation

You will find all relevant manuals and related documentation on the AccuLink Installation DVD.

- Insert the AccuLink Installation DVD in your DVD drive.
- Browse the DVD drive and select the folder **Manuals**.  
The available manuals are shown as pdf files.

**Note:** Make yourself familiar with AccuScreen, AccuLink, and their functions and settings, before you start testing.

## AccuLink installation

**Caution** - Do not connect the AccuScreen docking station to the PC before you have fully installed the software from the Installation DVD.

AccuLink is installed as a stand-alone program.

### Preparing to install AccuLink

To prepare your system for the installation of AccuLink, do the following:

- Make sure that you have sufficient memory and disk space (see the system requirements listed on the back cover of the box containing the AccuLink Installation Disk).

### Installing AccuLink

Install AccuLink from the AccuLink Installation DVD:

- Close all Windows programs to avoid conflict with the installer software.
- Insert the AccuLink Installation DVD in your DVD drive. The Installation Wizard will start up automatically. If it does not, select **Start > Run** and type **d:\acculink.exe**, where **d:** denotes your DVD drive, and **acculink.exe** is the command for executing the Installation Wizard.
- The Installation Wizard will guide you through the installation.
- Select the items in the order they appear and follow the instructions on the screen.
- Click on **Finish** to exit the Installation Wizard.
- You are now ready to connect AccuScreen. See the AccuScreen User Manual.
- When you have connected AccuScreen, launch AccuLink.
- You will be required to enter **User Name** and **Password**:  
**User Name:** Admin  
**Password:** 1234

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## Responsibility of the manufacturer

The manufacturer reserves the right to disclaim all responsibility for the operating safety, reliability and performance of equipment serviced or repaired by other parties.

## Copyright notice

No part of this Installation guide or program may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of Otometrics A/S.

## Copyright© 2013, Otometrics A/S

Printed in Denmark by Otometrics A/S, Denmark  
All information, illustrations, and specifications in this document are based on the latest product information available at the time of publication. Otometrics A/S reserves the right to make changes at any time without notice.

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### Copyright © 2013, Otometrics A/S
Tryckt i Danmark av Otometrics A/S, Danmark