

# OTOsuite and the CUNY Speech Test

User Guide

Doc no. 7-50-1300-EN/00 Part no. 7-50-13000-EN





No part of this documentation or program may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of GN Otometrics A/S.

Copyright© 2012, GN Otometrics A/S Published in Denmark by GN Otometrics A/S, Denmark

All information, illustrations, and specifications in this manual are based on the latest product information available at the time of publication. GN Otometrics A/S reserves the right to make changes at any time without notice.

# Version release date 10-10-2012

**Technical support** Please contact your supplier.

2 Otometrics - XXXX

# **Table of Contents**

1	Introduction	4
2	Unpacking and Installing	5
3	CUNY Testing	6
4	OTOsuite LIPread™ module safety	10

Otometrics - XXXX 3

# 1 Introduction

The OTOsuite CUNY speech test is designed to integrate the British English version of the CUNY speech material for play-back in the OTOsuite software.

The purpose of the CUNY test is to test a person's ability to understand speech with and without visual cues and noise.

The test consists of video presentations where the patient is prompted to repeat as much as possible of the sentences presented. The test can be presented with and without sound or video so that lipreading ability can be assessed for rehabilitation purposes.

When OTO suite is used as a NOAH module, all historical CUNY data is always available on the screen and in reports so that the patient's progress can be followed over time.

# 1.1 Intended use

The OTOsuite CUNY speech test is intended for use in connection with hearing habilitation/rehabilitation performed by audiologists, speech therapists and other trained personnel.

# 1.2 System overview

#### The OTOsuite CUNY speech test

- · Testing in OTO suite is supported by MADSEN Astera.
- OTOsuite with the module for performing the CUNY test is supplied on the OTOsuite Installation disk
- The CUNY speech material is supplied on a separate disk.

# 1.3 About this manual

This is your guide to using the OTOsuite CUNY speech test.

We strongly recommend that you read through the screen descriptions in this manual before you use the OTOsuite CUNY speech test for the first time.

**Note** • If you are using the OTOsuite CUNY speech test with Noah, we recommend that you are familiar with the screens and functions provided in Noah.

# 1.3.1 Safety

This manual contains information and warnings which must be followed to ensure the safe performance of the OTOsuite CUNY speech test.

Warning • Local government rules and regulations, if applicable, should be followed at all times.

Safety information is stated where it is relevant, and general safety aspects are described in OTOsuite LIPread™ module safety ▶ 10

# 1.4 Typographical conventions

# The use of Warning, Caution and Note

For safety reasons and appropriate use of the device and/or software, the manual contains **Warnings**, **Cautions** and **Notes**. These headings are used as follows:

Warning • Indicates that there is risk of death or serious injury to the user or patient.

Caution • Indicates that there is a risk of injury to the user or patient or risk of damage to data or the device.

Note • Indicates that you should take special notice.

# 1.4.1 Navigating this manual

Menus, icons and functions to select are shown in bold type, as for instance in:



Click the **Set options** icon on the toolbar or select **Tools > Options...** 

# 2 Unpacking and Installing

# 2.1 Unpacking

- 1. Inspect the package and its contents for possible visual damage.
- 2. Check with the packing list to make sure that you have received all necessary parts and accessories. If your package is incomplete, contact your supplier.

# 2.2 Installation

For instructions on installing OTOsuite, see the OTOsuite Installation Guide which is inserted in the OTOsuite Installation box, and the OTOsuite User Manual.

Whether you install OTOsuite or the CUNY speech material first is of no consequence.

- 1. Install OTOsuite.
- 2. Install the CUNY speech material supplied on a separate disk.
- 3. Enter the CUNY license key in the OTOsuite Configuration Wizard. Select **Tools > Configuration Wizard...** and click the **Configure** button next to **Audiometry**.
- 4. Click **Next** until you reach the screen for entering license keys and enter the license key.
- 5. If the **LIPread** tab is not shown in OTOsuite, enable viewing the tab in the **Configuration Wizard**.

# 3 **CUNY Testing**

# 3.1 Navigation

For general information about navigating in the test screen, see also the OTOsuite User Guide.

The main screen consists of

- a Scoring Counter at the top of the screen,
- a Player Panel for selecting speech lists and controlling the test,
- a List View for viewing and scoring in the selected list during presentation,
- a Results Table for viewing and comparing the results.

# 3.1.1 The Scoring Counter

8/132 80% List 1

During playback and scoring, the **Scoring Counter** at the top of the screen shows

- the number of words in the list scored correctly of the total number of words in the list
- the percentage of correctly scored words
- the list currently being presented.


# 3.1.2 The Player Panel



The **Player Panel** is a group of buttons enabling you to control the test. You can select the list for the test, and you can play, pause, resume, and stop the test.

Player Panel			
<b>•</b>	Play During the test:  the List View shows only the speech list that has been selected for the current test.  you can click on any sentence to jump freely in the selected speech list.		
	<ul> <li>click the Play button to play back the selected sentence.</li> <li>Stop</li> </ul>		
• Store	Stops play-back of the current sentence.		
Terminates the test and stores the result.      Reset  Terminates the test without storing any results.			
List selection drop-d  Click the drop-down	own list to select which speech list to use in the test.		

## 3.1.3 The List View

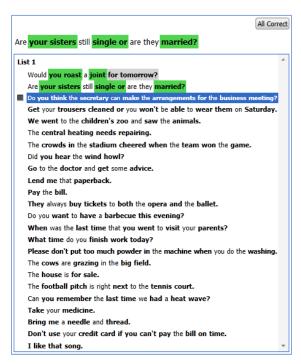

When you start play-back of the selected speech list, the **List View** shows the selected lists.

Each sentence in a list has a number of highlighted key words that can be scored as **Correct** or **Incorrect**.

Words that can be scored are displayed in **bold type**.

The sentence being presented is shown at the top of the list. This is where you score correctly repeated words by clicking on them.

- When you score a word as Correct, it is highlighted in green.
- When you score a word as Incorrect, it is highlighted in grey.

#### **Practice List**

For practicing the test flow so that the client is familiar with the procedure before the actual test starts. Results from a practice list cannot be stored.

## **During play-back**

An icon to the left of the sentence that is being presented shows the status.



Indicates that the list is being presented.



Indicates that the list is stopped.

## **Scoring**

The key words are scored after play-back of each sentence. The key words to be scored are shown in BOLD type.

You can change a previous score by right-clicking the word and selecting **Change Score**.

# 3.1.4 The Results table



During review (when the Player is in idle mode), the **List View** shows all lists that have been included in the **Results** table.

You can add a comment of your own choice next to each test.

Results Table	Results Table		
Ear	Shows which ear was tested.		
%	Indicates the correct score.		
dB	Shows the stimulus and masking levels used in the test.		
List	Shows the number of the tested list.		
Info	Displays the test conditions, for example whether sound was muted or video was included, etc.		
Date and Comments	Displays the date of the test and any comments you may have entered.		

# 3.2 Preparing for CUNY testing

# 3.2.1 Preparing the patient

You can use a practice lists to familiarize the patient with the test protocol.

1. Instruct the patient to repeat the sentences that are presented during the test.

# 3.2.2 Preparing OTOsuite

- 1. Click on the LIPread tab in the Audiometry module.
- 2. Set up the test to suit your purposes: select **Tools > Options > Audiometry**.

 You can select specific screens to prepare the patient for the next presentation.



- Displaying an indication of the topic of the sentence (e.g. food).
   The message is shown in order to add a contextual help for the patient.
- You can disable sound in order to assess the patient's ability to lipread.




- 3. Select the speech list
  - The list can be presented either through earphones or sound field loudspeakers.
- 4. Click the **List Selection** button to expand and select the list you wish to include in the test.



5. Test monaurally or binaurally. Insert earphones are recommended for monaural testing. Their high interaural attenuation typically makes masking of the non-test ear unnecessary.

# 3.3 Testing

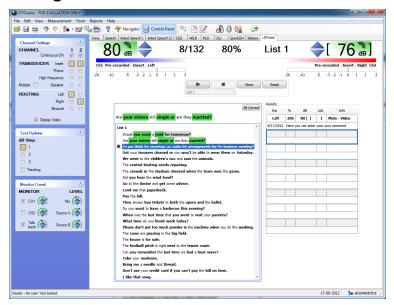

1. Select the desired test list in the player.

- 2. Present the test with earphones or in a sound field.
- 3. Instruct the patient in how to respond to the presented material.
- Score the key words highlighted in each sentence by clicking the words on the screen. You can also enter a full sentence score by clicking the All Correct button.
- 5. You can store results in the **Results Table** by clicking the **Store** button.

# 4 OTOsuite LIPread™ module safety

This manual contains information and warnings, which must be followed to ensure the safe performance of the devices and software covered by this manual. Local government rules and regulations, if applicable, should also be followed at all times.

When the OTOsuite LIPread™ module is used in conjunction with a test device, make sure that all information and warnings in the User's Manual of the test device are followed.

For safety specifics concerning test modules and test devices, see the specific manuals.

# 4.1 Symbols used



Complies with Medical Devices Directive 93/42/EEC.



Used in error message dialogs if software program fails. See the detailed information in the dialog box.

# 4.2 Manufacturer

GN Otometrics A/S 9 Hoerskaetten, DK-2630 Taastrup Denmark

+45 45 75 55 55 +45 45 75 55 59

www.otometrics.com

## 4.2.1 Responsibility of the manufacturer

The manufacturer is to be considered responsible for effects on safety, reliability, and performance of the equipment only if:

- All assembly operations, extensions, re-adjustments, modifications or repairs are carried out by the equipment manufacturer or personnel authorized by the manufacturer.
- The electrical installation to which the equipment is connected complies with EN/IEC requirements.
- The equipment is used in accordance with the instructions for use.

The manufacturer reserves the right to disclaim all responsibility for the operating safety, reliability and performance of equipment serviced or repaired by other parties.