


Document Description

The document describes how to configure the Aurical PMM.


1. Open OTOsuite.



2. Go to **“Tools”** and the **“Configuration Wizard”** in the OTOsuite menu.

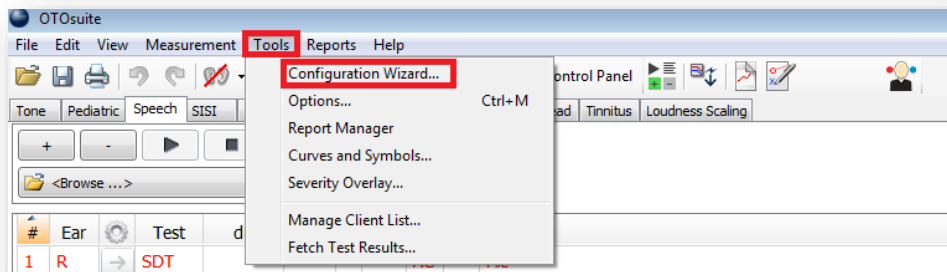


3. Make sure the check box next to PMM is selected. Click **“PMM configure”**.

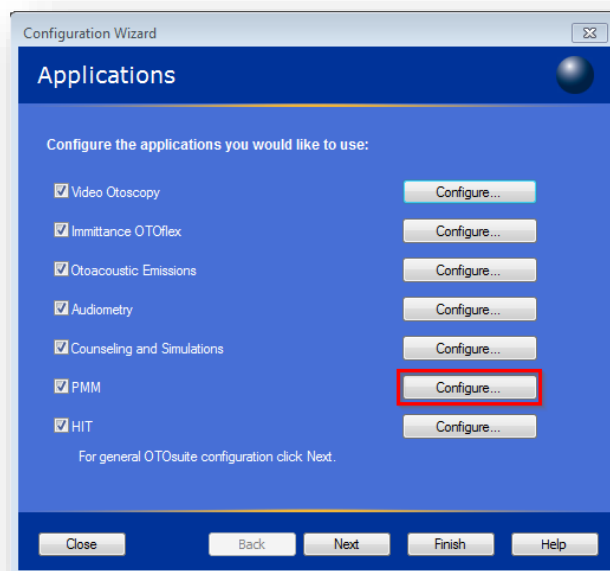


4. Switch on the FreeFit and select **“My device is turned on and ready to be found”**. Select the **“Use only**


OTOair for device connection” and click on “Search”.

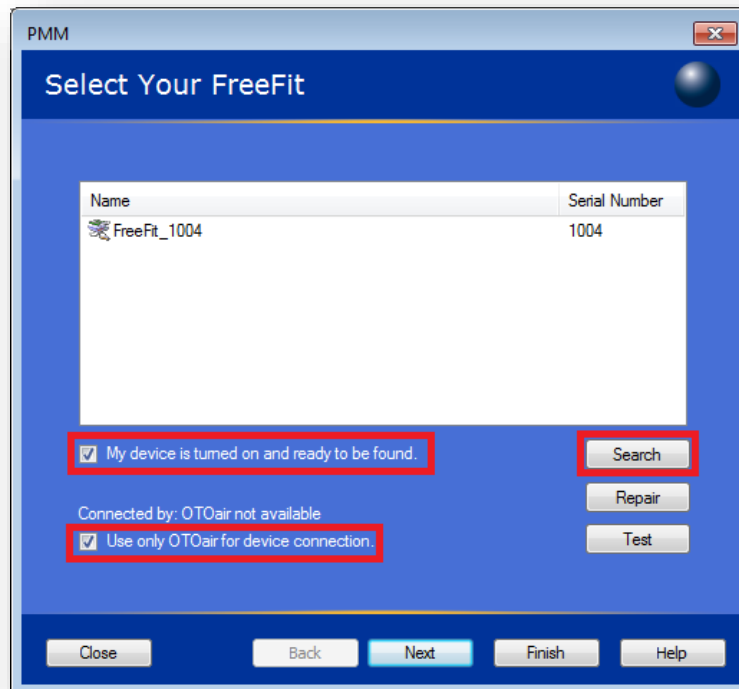


5. FreeFit searching in process.

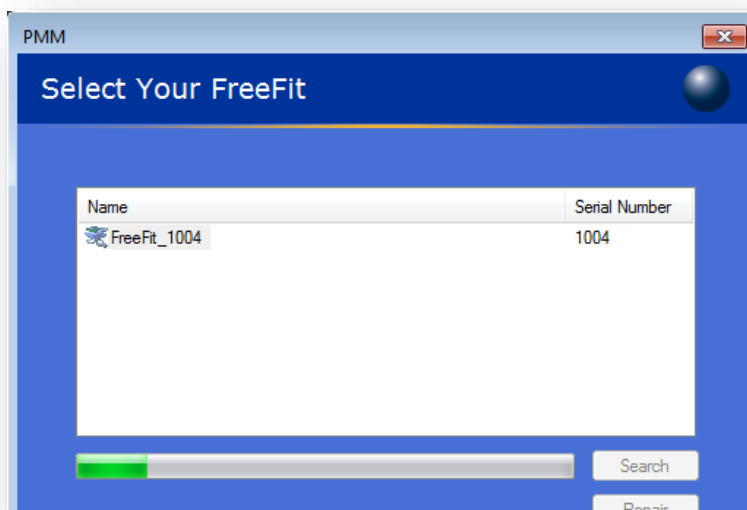



6. Select the FreeFit S/N that you are trying to connect (highlighted in blue) and click **“Next”**.

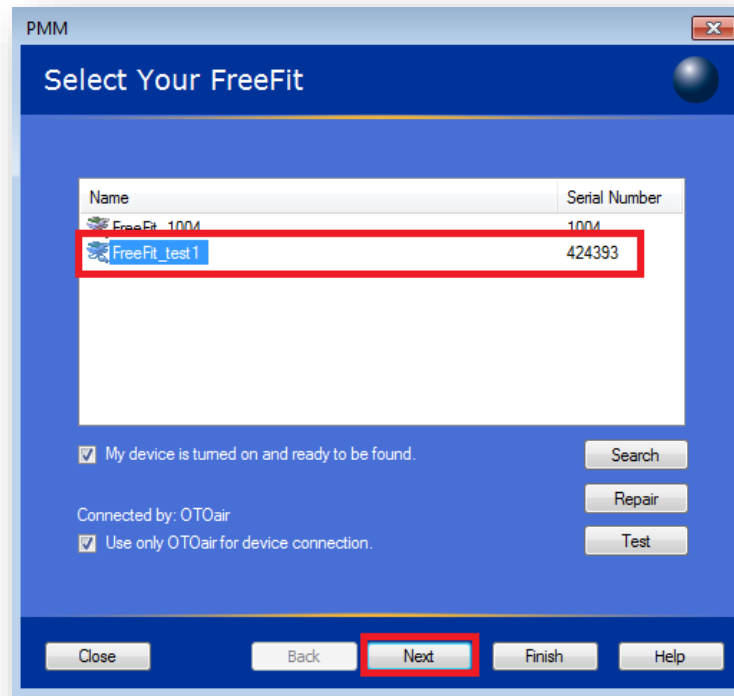


7. Select the Aurical Aud as Soundcard for the **“Microphone”** input, **“Speaker output”** and the **“Headphone output”**. (Speaker channel Front and Rear is normally not changed)

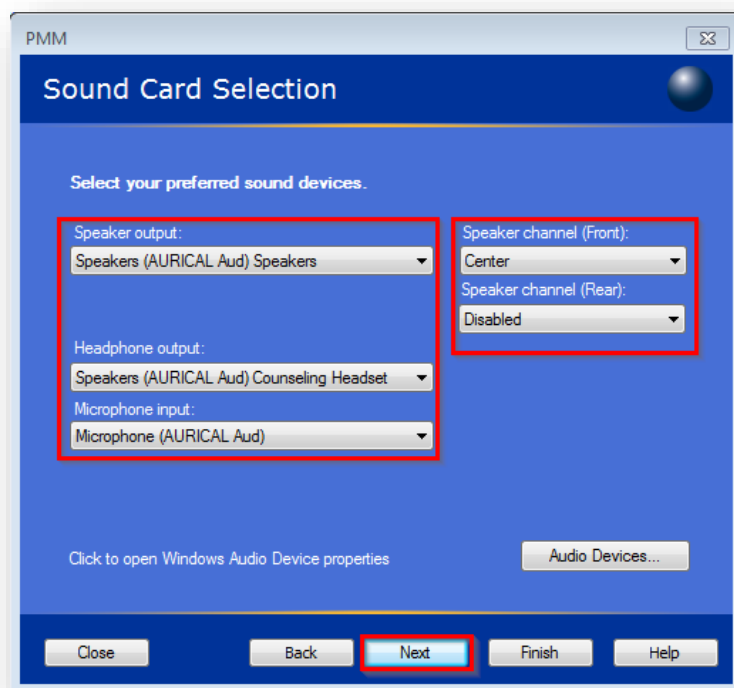


8. Select the various **“Test Types”** that you would like to be shown in the OTOsuite toolbar. Click **“Next”**.

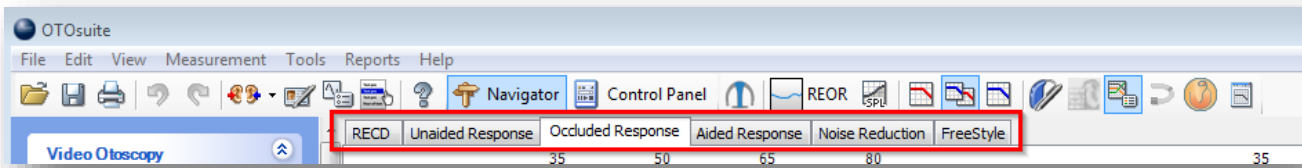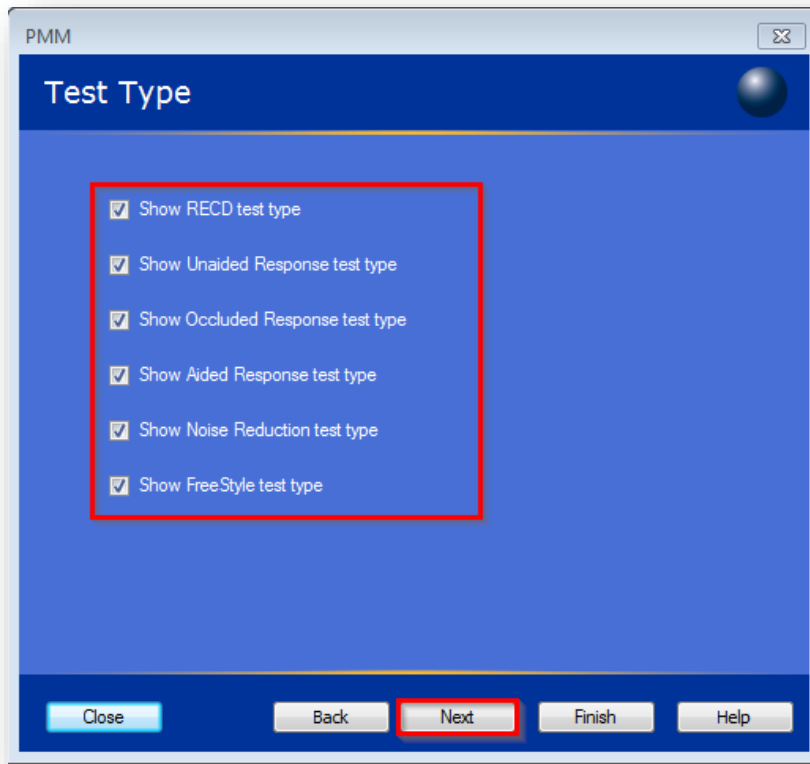

9. Keep the RECD “**Always make both the Ear and Coupler measurement**” un-checked.

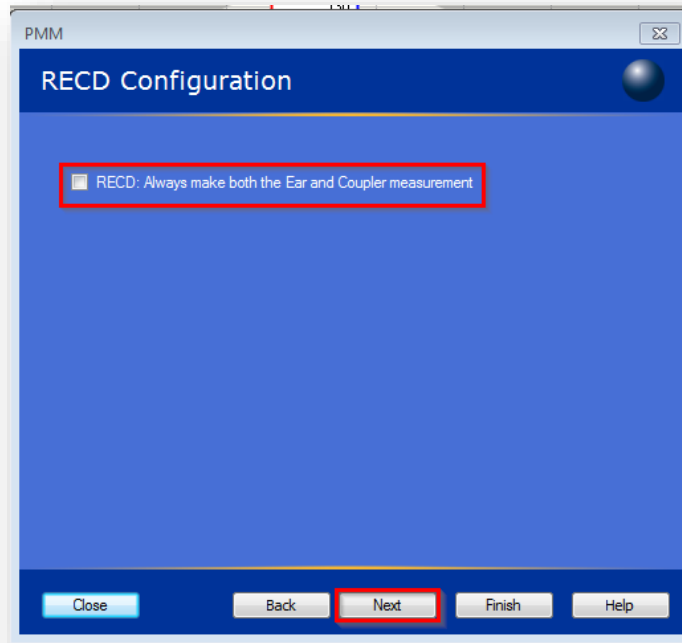


10. For a stationary installation select the “**Monthly**” calibration frequency. If the Aurical PMM is not used stationary, select the “**Daily**” calibration.

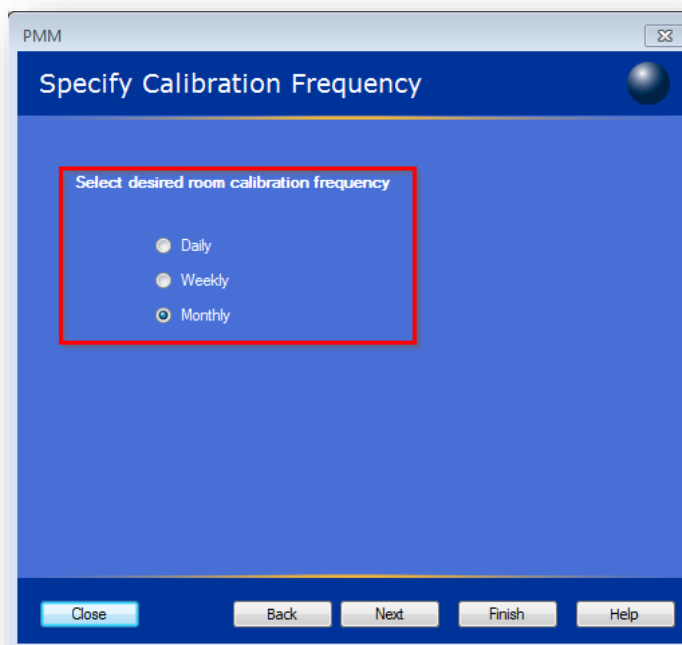


11. Place the FreeFit in the position where the patient is sitting. Select the box **“My FreeFit is turned on and within reach”**. Click on **“Next”**


The Note is not to be taken into consideration when using the Aurical PMM.

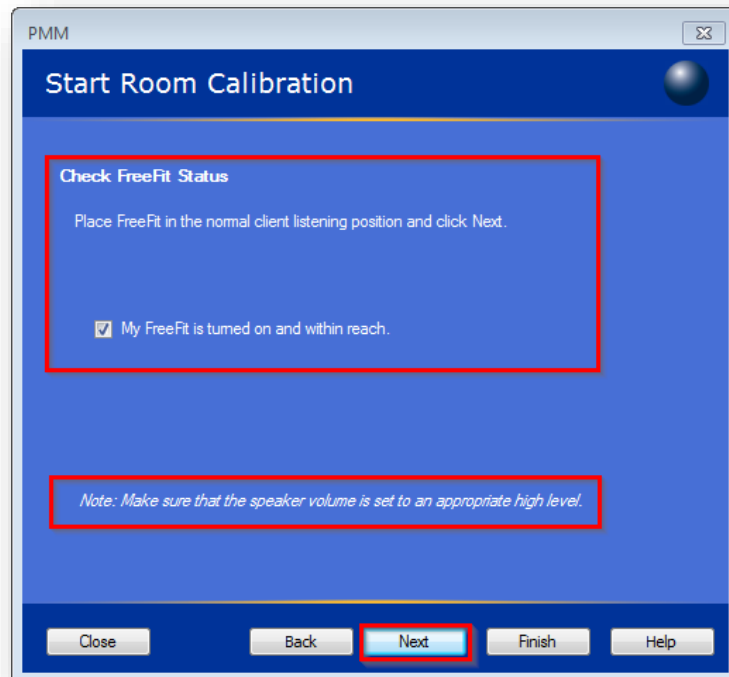

12. Room Calibration in process. You will hear a sound play out of the AURICAL PMM speaker.

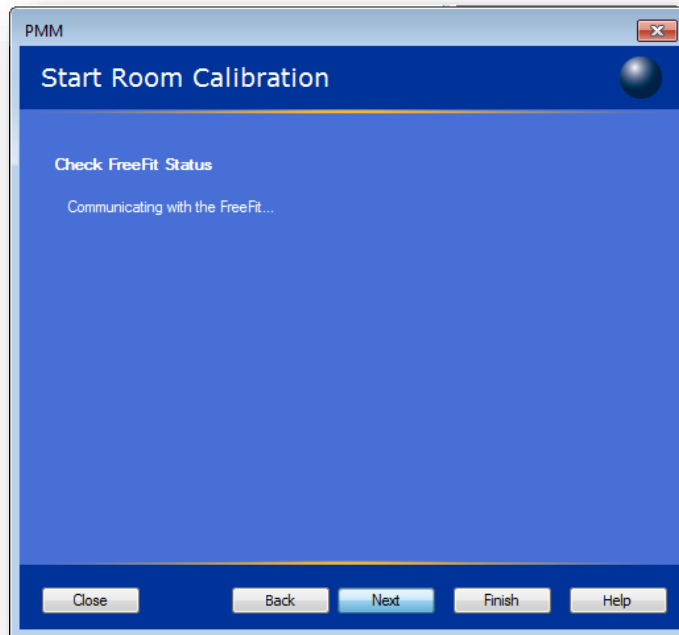

13. Room Calibration has finished successfully. Click Finish to finalize the configuration

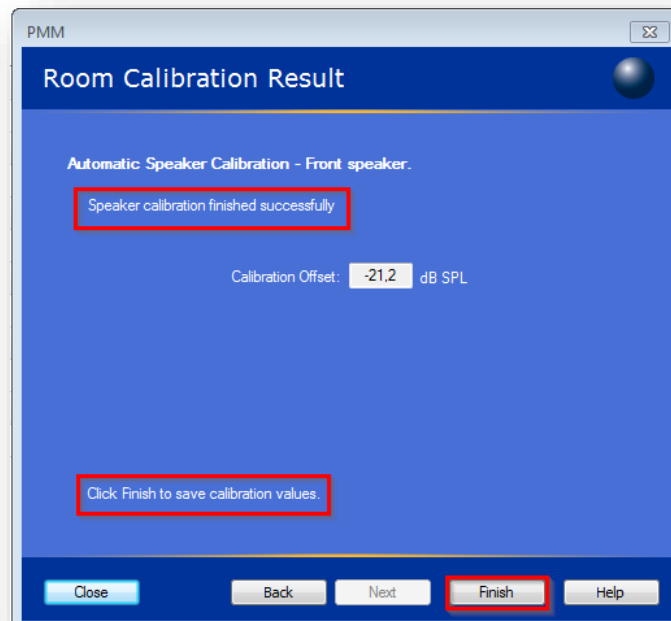

14. Click on Close to exit out of the configuration.

