


Document Description

The document describes how to verify the current Madsen Accuscreen firmware and perform an update.


1. Verify the installed firmware version by clicking from the Home page on **“Settings” \ “System info”**




2. Update the Acculink software with the current version.
3. To do the firmware update **“Log in”** to the Acculink application, and make sure that the Madsen Accuscreen is placed in the Powered Charger and is turned on.

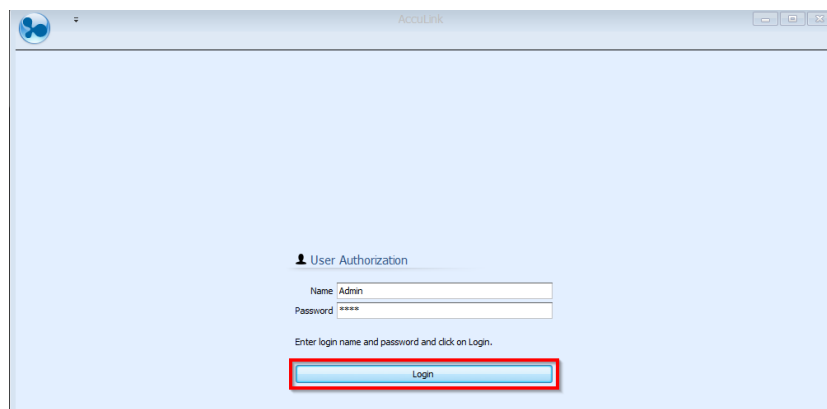


4. Click on the **“Update”** icon.

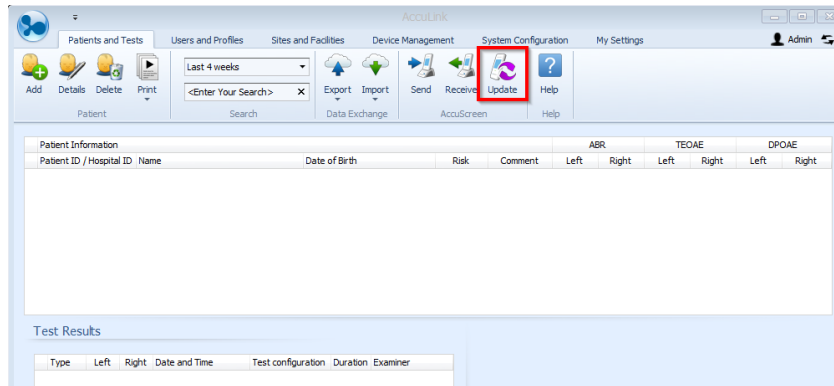


5. The connected Accuscreen will now be visible in the Configuration Assistant window. Highlight the instrument and click **“Next”**.

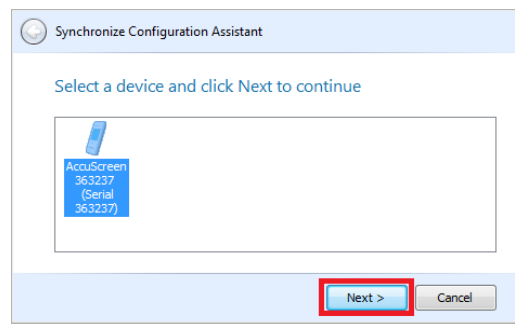

6. The Firmware update is in progress... It's very important to leave the device in the Docking station during this update .

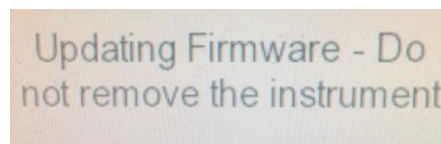

7. The firmware update is now finalized. Click **“Finish”**.

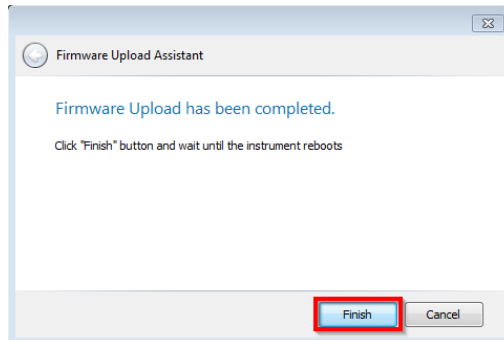

8. The Madsen Accuscreen will now reboot, and start up in the Home screen.



9. Now verify that the firmware is updated correctly with a higher version number.

