

Document Description

The document describes how to install US Wordlist / Speech material, 8-49-82400_x from either a DVD or a Zipped file.


Installation from the DVD:




1. Insert the DVD into the Computer drive.
2. If the DVD content doesn't automatically open up, browse to the DVD drive using Windows explorer on the PC.

Installation from a Zip file:


3. Right click on the Zip file and select **"Extract All"**

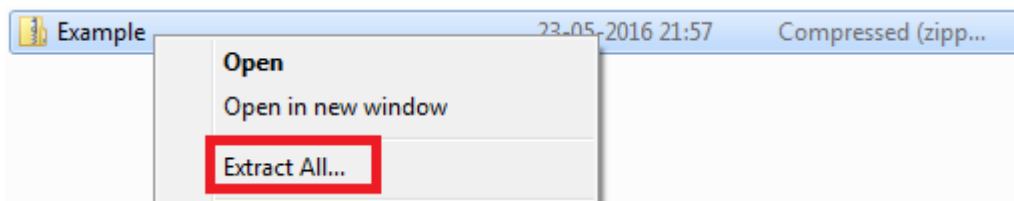


4. Make sure that the **“Show extracted files when complete”** is selected, and click the **“Extract”** button.

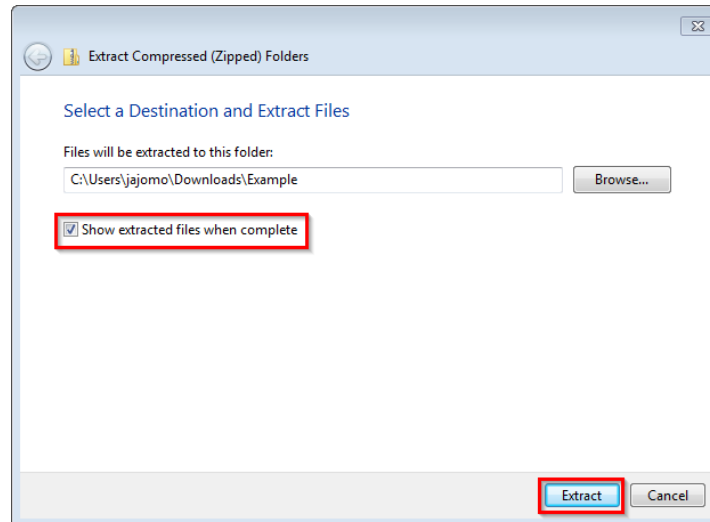


5. Extracting the Zip file is in progress.

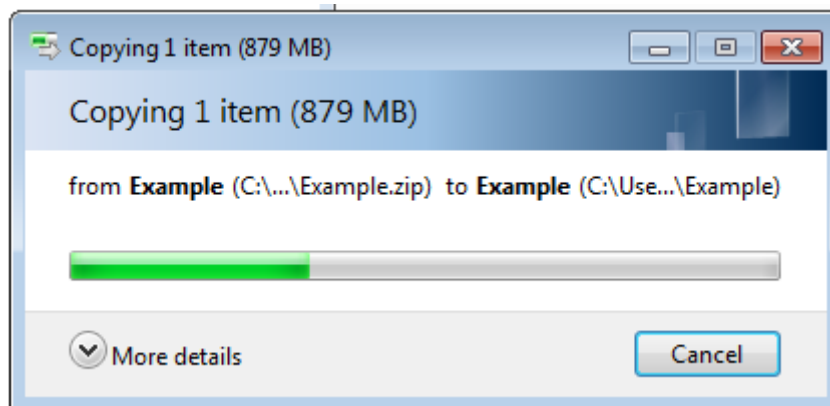



6. Click on the **“OTOsuite Wordlists - US”** file to begin the Wordlist installation.

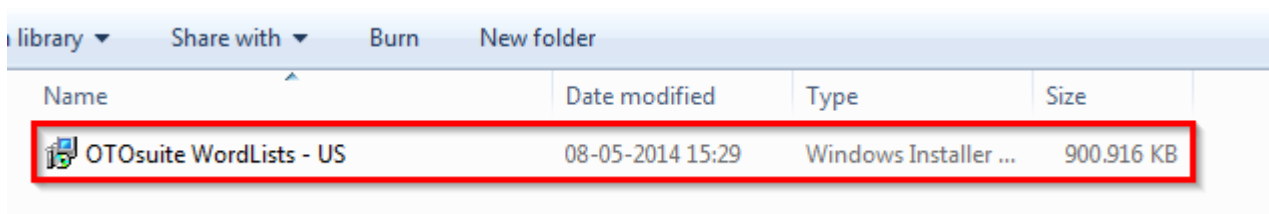


The screenshot shows a Windows File Explorer window. The top bar includes 'library', 'Share with', 'Burn', and 'New folder'. Below the top bar is a table with columns for 'Name', 'Date modified', 'Type', and 'Size'. The table contains one row with the following data:

Name	Date modified	Type	Size
 OTOsuite WordLists - US	08-05-2014 15:29	Windows Installer ...	900.916 KB


The row containing the file 'OTOsuite WordLists - US' is highlighted with a red box.

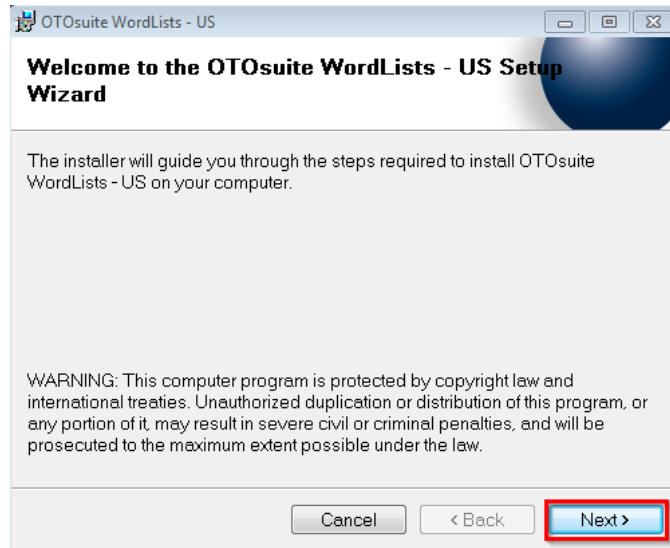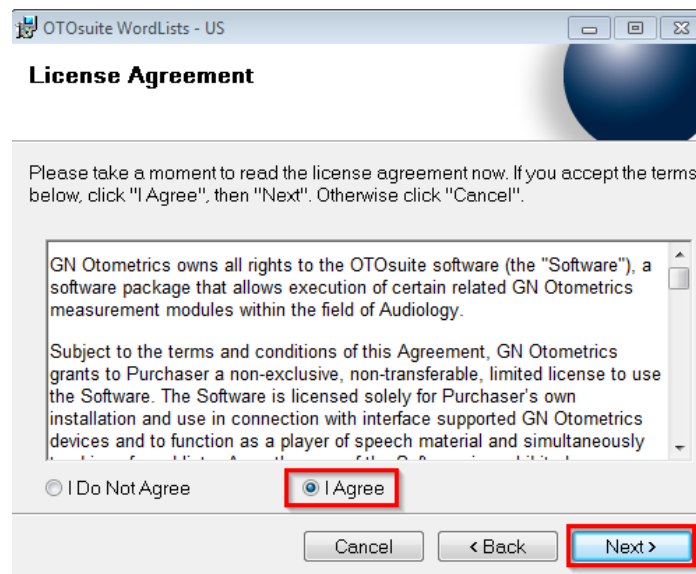7. Click **“Next”**8. Mark the **“I agree”** check box, and click **“Next”**


9. Select the speech material to install, and click **“Next”**

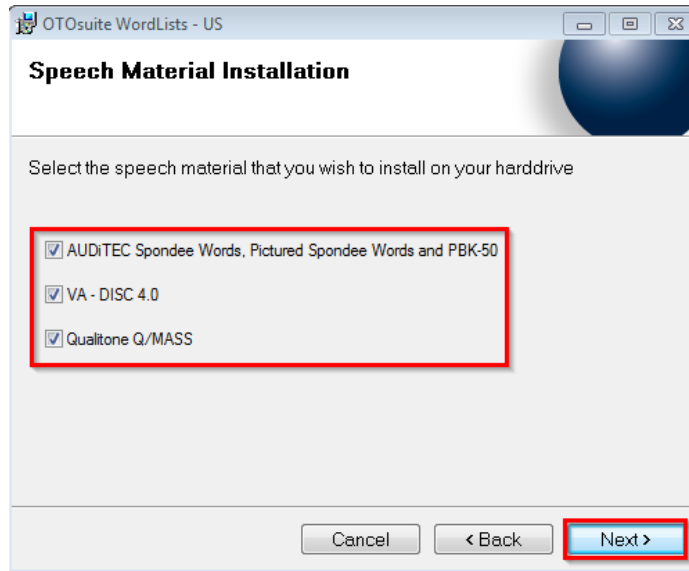


10. Click **“Next”** to begin the installation.

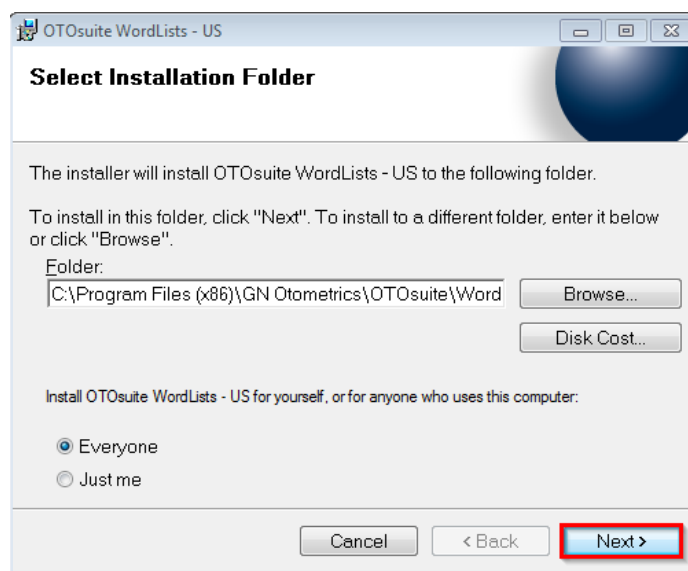


11. Click **“Next”** to confirm and start the installation.

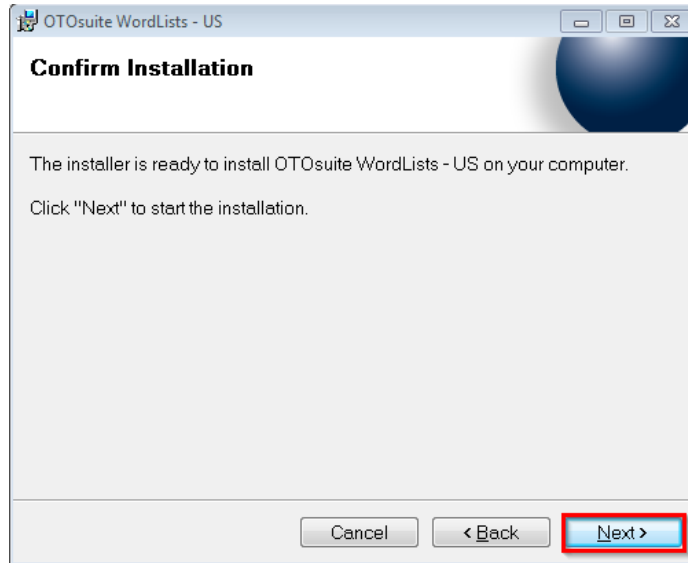

12. Installation in process, if the yellow UAC icon shows up in taskbar, click on it and select **“Yes”**.

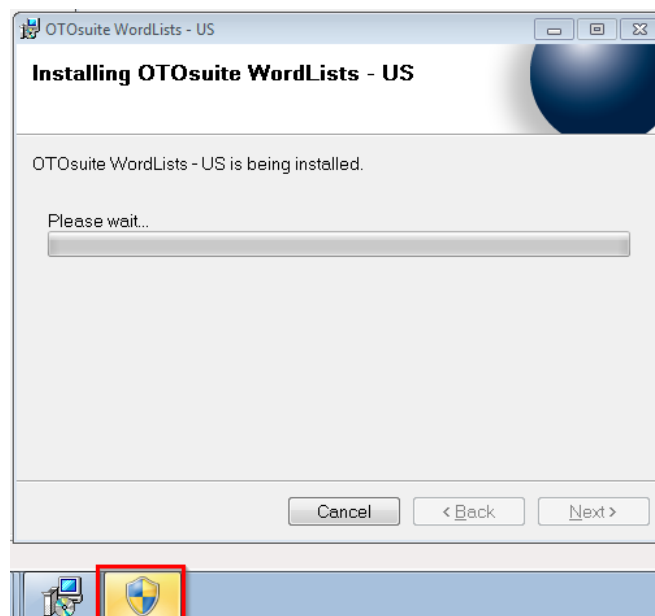

13. Installation will continue.

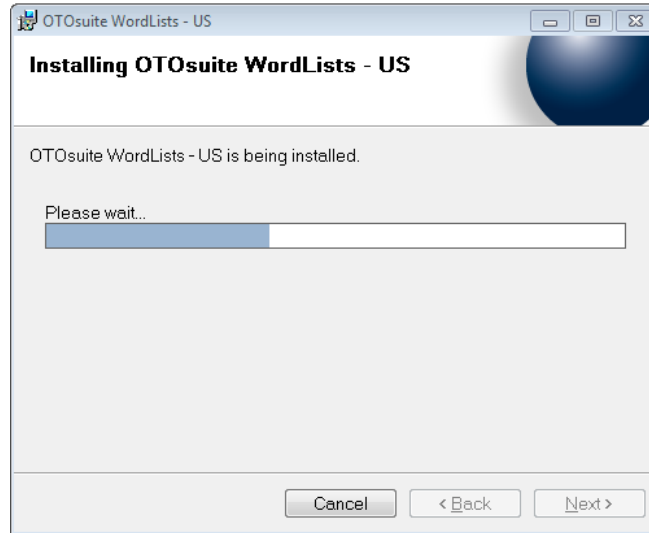

14. Installation is complete. Click **“Close”**

