

## Document Description

The document describes how to install OTOsuite PC software from either a DVD or a Zipped file.


### Installation from the DVD:




1. Insert the DVD into the Computer drive.
2. If the DVD content doesn't automatically open up, browse to the DVD drive using Windows explorer on the PC.

### Installation from a Zip file:


1. Right click on the Zip file and select **"Extract All"**

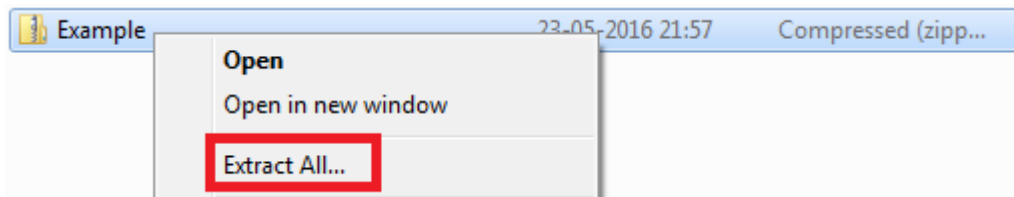



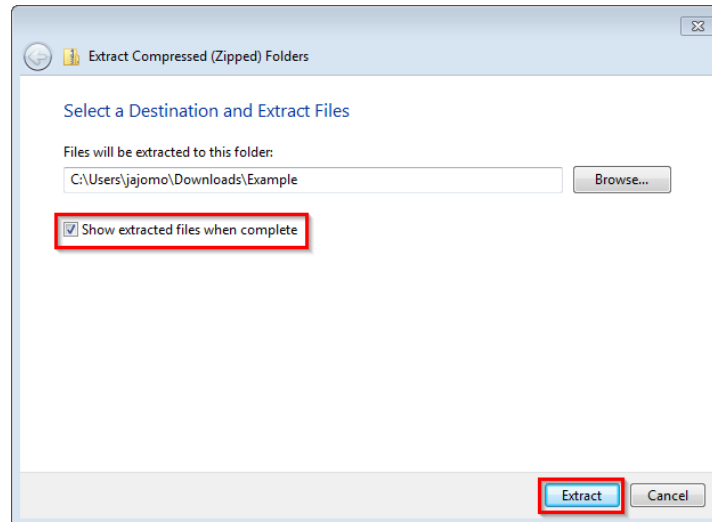

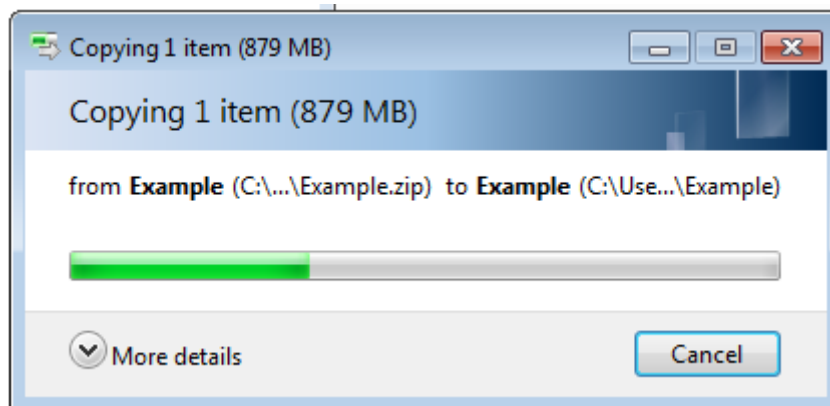


2. Make sure that the **“Show extracted files when complete”** is selected, and click the **“Extract”** button.




3. Extracting the Zip file is in progress.




4. Click on the **“setup”** file to begin the OTOsuite installation.

 Installation - Japanese	11-03-2016 05:21	Compiled HTML ...	514 KB
 Installation - Svensk	11-03-2016 05:23	Compiled HTML ...	313 KB
 Installazione - Italiano	11-03-2016 05:21	Compiled HTML ...	314 KB
 Kurulum - Türk	11-03-2016 05:23	Compiled HTML ...	311 KB
 OTOsuite	29-03-2016 14:01	Adobe Acrobat D...	1 KB
 <b>setup</b>	29-03-2016 14:03	Application	1.135.079 KB
 Telepítés - Magyar	11-03-2016 05:21	Compiled HTML ...	312 KB


5. The following screen may populate if required Windows updates are needed. Click **“Install”**.

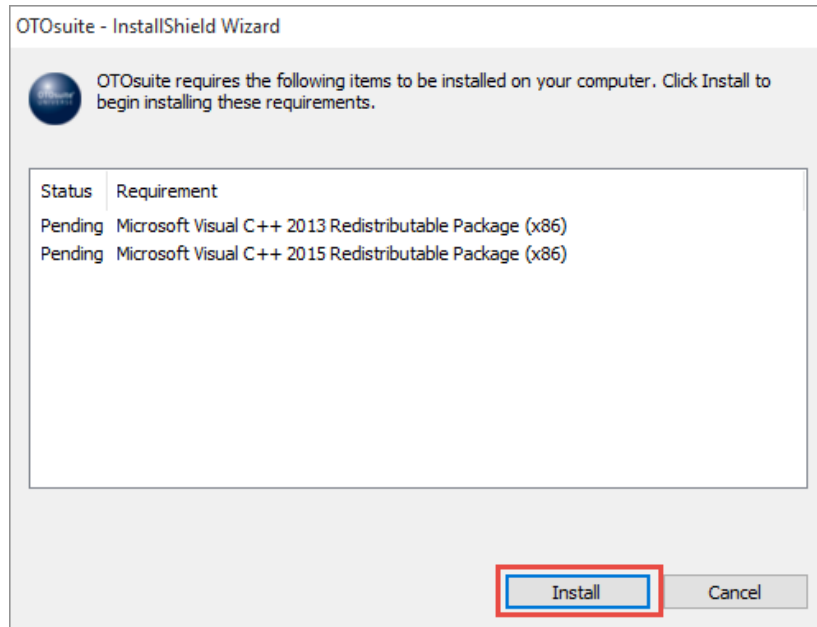


6. Select **“Next”**.

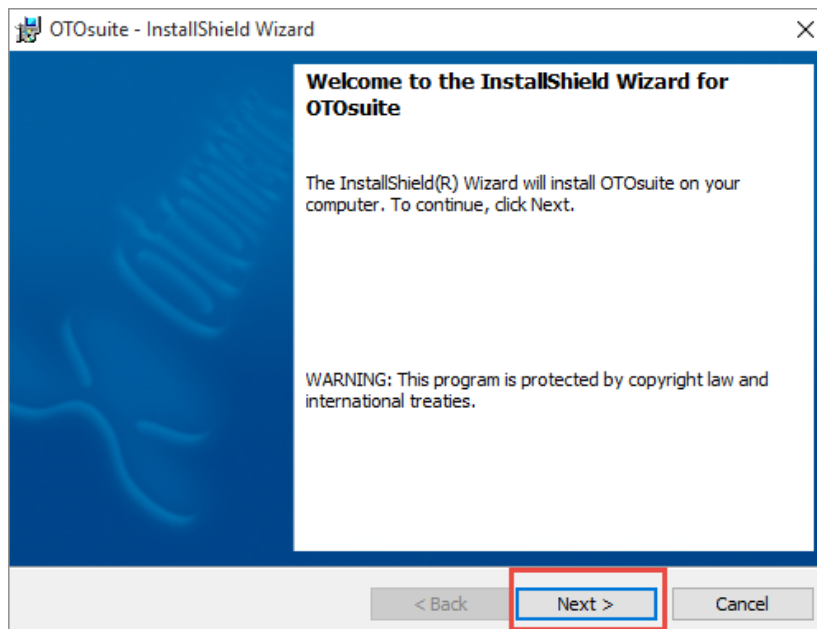


## 7. Select “Next”.

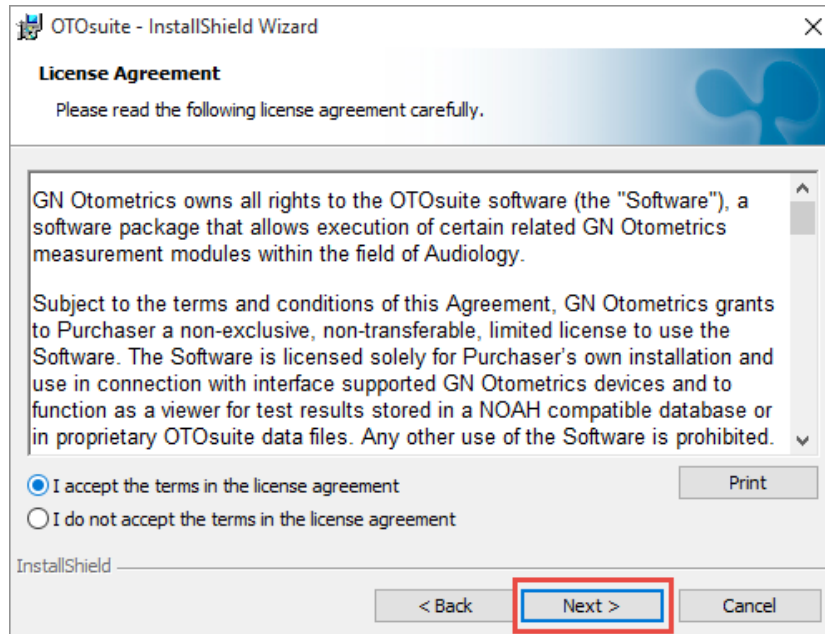


## 8. Select “Next”

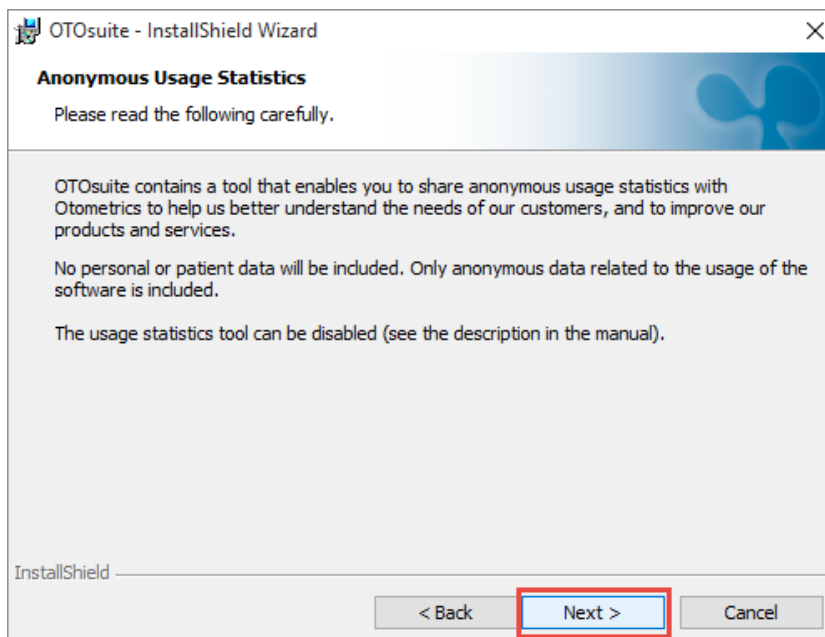


9. Select the “Install as a single module in NOAH” and Click “Next”.

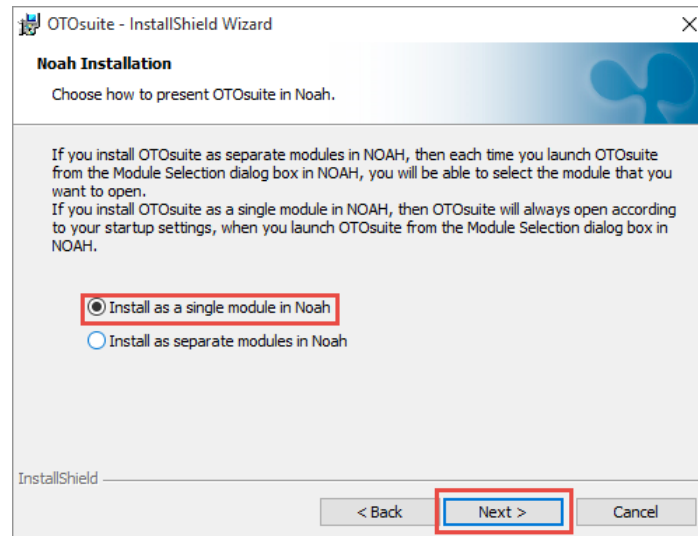


10. Click “Next”.

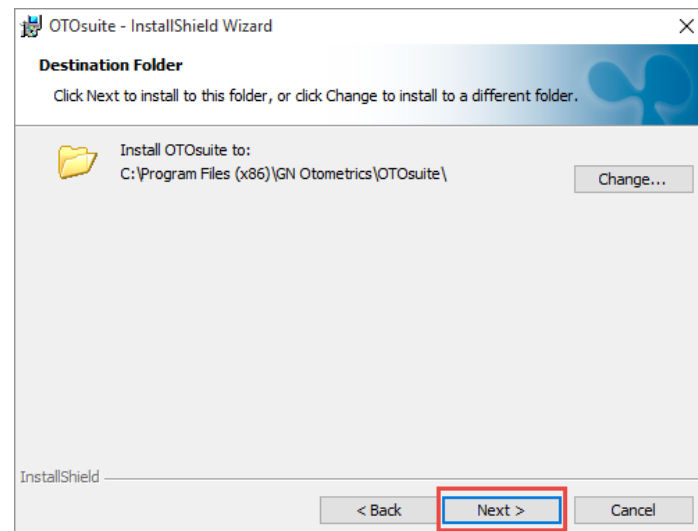

11. Click **“Install”**.

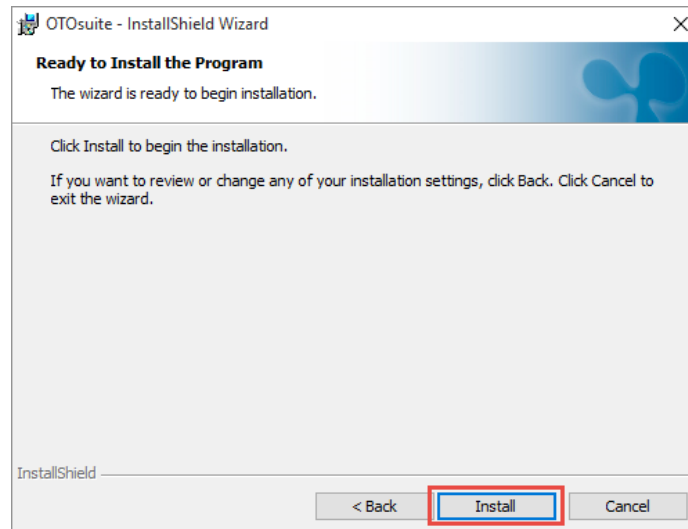

12. Once installed click **“Finish”**.

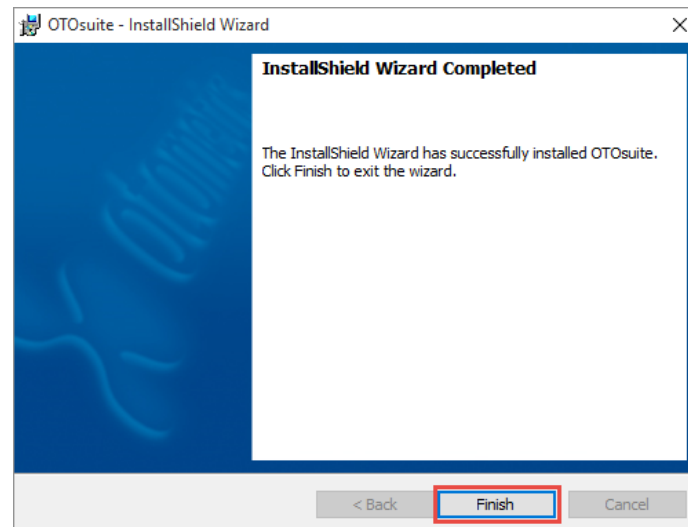

13. OTOSuite is now available from NOAH, or as standalone.

